



Human Resonance Wave  Infra-red Radiator Aladdin·H[®]

www.aladdin-h.com

 Aladdin·H[®]

Promotion of Health Search for Happiness

TAERIM MEDICAL 

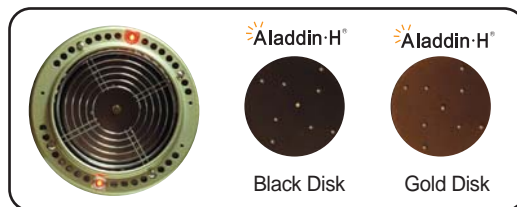
TAERIM MEDICAL 

#108 Hyundai I-Vally, 223-12 Sangdaewon-dong, Jungwon-gu, Seongnam-si, Gyeonggi-do, Korea

Tel: 82-31-737-9181 Fax: 82-31-737-9182 E-mail: taerim@aladdin-h.com <http://www.aladdin-h.com>

Aladdin·H[®]

Product information (For Hospital)



We invented our products which have different models to be used by various.



TR-2000

Obstetrics/Gynecology/Urology



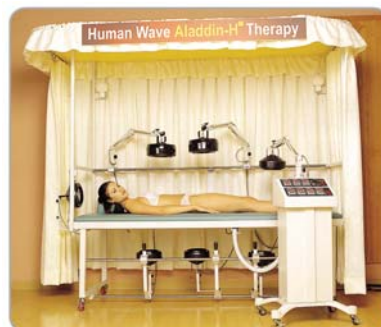
DS-777D/TR-2000D

Medical Clinics



DS-777S/TR-1000S

Medical Clinics



TR-7000

(for whole body treatment)
General Hospitals

TABLE OF CONTENTS

1. Human Resonance Wave Aladdin-H

| | |
|--------------------------------------|---|
| Human Wave Therapy | 2 |
| Human Resonance Wave Aladdin-H | 3 |
| How Aladdin-H Works? | 4 |
| Characteristics of Aladdin-H | 5 |
| Major use and effects | 6 |

2. Direction for use

| | |
|---------------------|----|
| How to cure | 7 |
| Attention | 9 |
| Warning | 10 |
| Healing Point | 11 |

3. Direction for operation

| | |
|----------------|----|
| TR-300 | 34 |
| DS-777 | 36 |
| TR-1000 | 38 |
| TR-2000 | 40 |
| TR-2000D | 42 |
| TR-7000 | 44 |

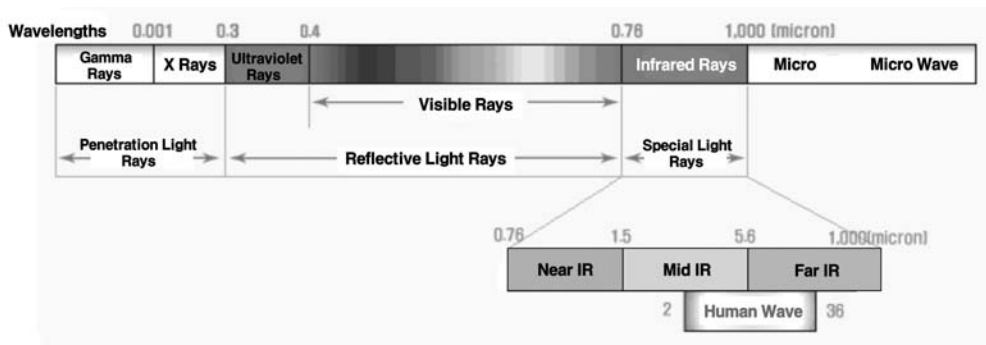
- Things to check prior to requesting A/S
- Warranty

Human Wave Therapy

Human Wave

- The wavelength range emitted from human body is 2-36 micron which is within the wavelength of infrared wave (0.76-1,000 micron). It is the necessary wave that human body takes in and out to maintain vitality.
- The major wavelength required in maintaining molecular motion and biochemical reaction is 6-10 micron.
- The wavelength required to maintain the body temperature of 36.5 °C is 5-15 micron.

Continuum of light and position



Human Wave Therapy

Human Wave Therapy has seven conditions of ideal waves essential to prevent and cure diseases.

Seven conditions of ideal waves:

1. There should not be any side effects caused by waves
2. Within wavelength range of human wave, 2-36 micron, therapeutic wavelength should range 5 - 15 micron with major wavelength range of 6-10 micron and be concentrated.
3. Waves should be able to penetrate deep in organs.
4. By supplementing to weakened human wave, the waves should promote cellular molecular motion in order to accelerate the process of regeneration of degenerated tissue and prevent aging.
5. Waves should promote circulation of vital energy and blood in order to facilitate provision of oxygen and nutrition to cells.
6. Waves should improve immunity, pacify nerves and vibrate.
7. Waves should control secretion of hormones and density of elements constituting human body.

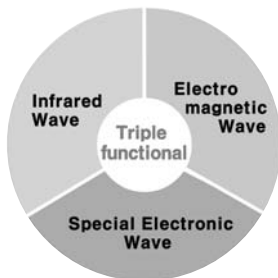
Ideal waves have above features. It is one of the essential tasks for modern medical science to develop such waves. Human Wave Therapy is the curative means which have all these features

What is Human Resonance Wave [⚡]Aladdin·H[®]



[⚡]Aladdin·H[®] is the medical device that radiates Human Resonance Wave which promotes blood circulation and secretion of Hormones, controls consistency of elements of human body, and alleviates pain and inflammation thereby shortening time of medical treatment for both acute and chronic disease as well as hard-to-cure diseases.

[⚡]Aladdin·H[®] generates triple medical functional waves simultaneously



- Infrared Wave - Thermotherapy
- Electro Magnetic Wave - Magnetic Therapy
- Special Electronic Wave - Acupuncture Therapy

The disk of [⚡]Aladdin·H[®] consisted of more than 30 kinds of elements of human body radiates waves

- have the same wavelengths as human resonance wave
- have superior magnetic force due to resonance between the same wavelengths
- have acupunctural effect by stimulating 60 times a second
- have no side effects

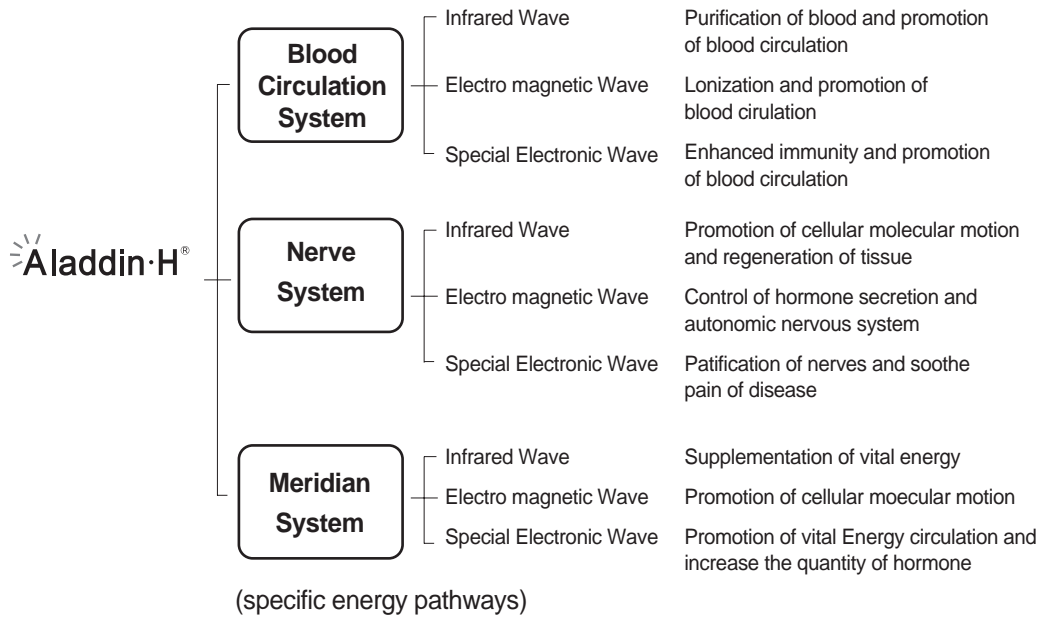
Comparison between Human Wave and [⚡]Aladdin·H[®]

| | Wavelengths | Major Wavelengths | Comparison |
|---|-------------|-------------------|------------------------------------|
| Human Wave | 2~36 | 6~10 | There is no significant difference |
| Human Wave Therapy [⚡] Aladdin·H [®] | 2~25 | 8~10 | |

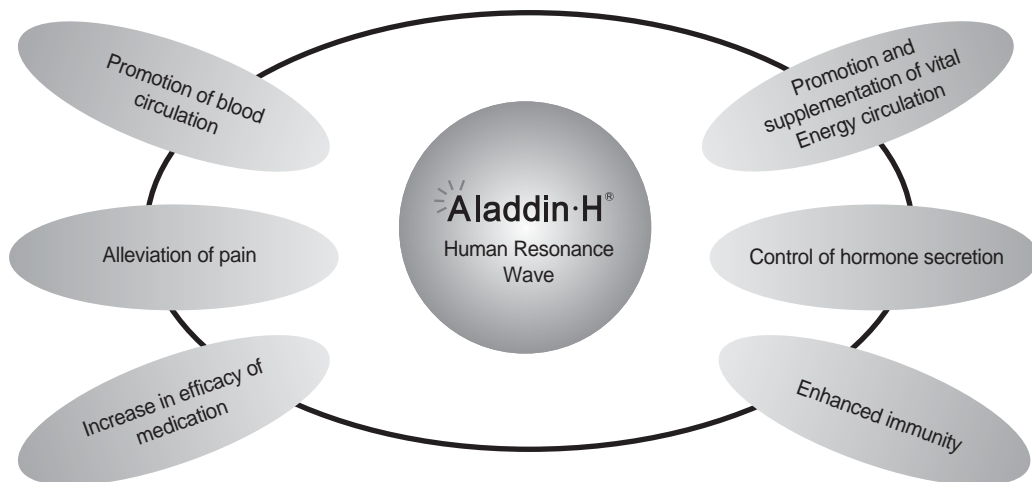
* Major Wavelengths - Wavelengths which make molecular motion of cells, Energy in the body and biochemistry operation. Most of Human Resonance operation Wave.

How Aladdin·H® works?

Major effects of Human Resonance Wave Aladdin·H®

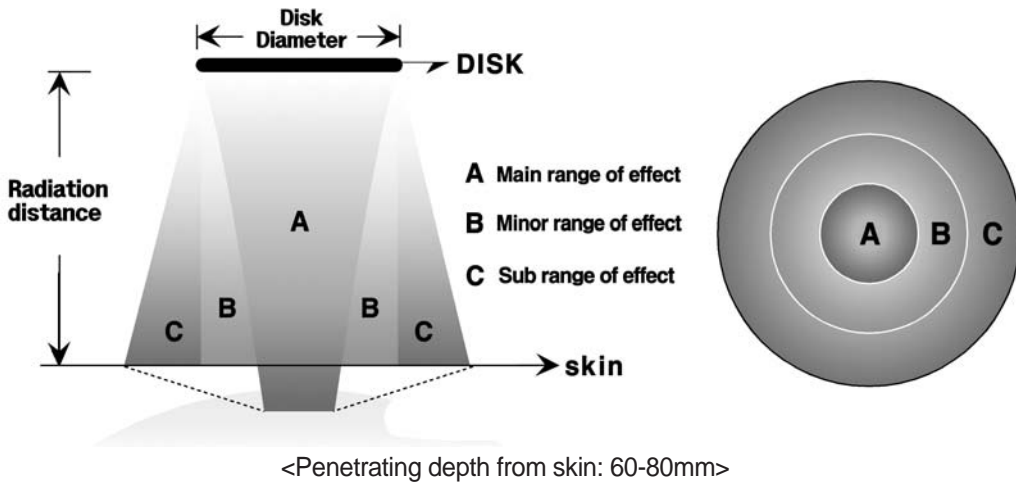


Synthesis effects of Human Resonance Wave Aladdin·H®



• Characteristics of $\text{Aladdin}\cdot\text{H}^{\circ}$

(1) Plane figure of Radiation and Range of effects



(2) Averages of treatment effect range, skin temperature and magnetic force

| S type (102mm) | | | | | G type (124mm) | | | | | |
|-------------------------|---------------------------------|-----|-----|-----------------------|-------------------------|---------------------------------|-----|-----|-----------------------|----------------|
| Radiation Distance (mm) | Diameter of Radiation Range(mm) | | | Skin Temperature (°C) | Radiation Distance (mm) | Diameter of Radiation Range(mm) | | | Skin Temperature (°C) | Magnetic Force |
| | A | B | C | | | A | B | C | | |
| 200 | 58 | 102 | 152 | 40 | 250 | 70 | 124 | 188 | 42 | 1.901 |
| 250 | 46 | 102 | 169 | 38 | 300 | 58 | 124 | 202 | 40 | 845 |
| 300 | 35 | 102 | 178 | 36 | 350 | 47 | 124 | 216 | 38 | 475 |

(3) Comparison of normal infrared rays and $\text{Aladdin}\cdot\text{H}^{\circ}$ infrared Rays(IR)

| Wavelength range | | Penetrating Depth | |
|---|--|--|--|
| $\text{Aladdin}\cdot\text{H}^{\circ}$ | Normal Infra Red | $\text{Aladdin}\cdot\text{H}^{\circ}$ | Normal Infra Red |
| <ul style="list-style-type: none"> - 2-25 micron, - Major wavelength : 9 micron - Almost identical with human wave. - Long radiation possible | <ul style="list-style-type: none"> - Varied wavelength range - Major wavelength exceeds human wave range. - Long radiation causes side effects. | <p style="text-align: center;">60-80mm</p> <p style="text-align: center;">(Can be used for direct treatment to internal organs)</p> | <p style="text-align: center;">10~20mm</p> |

• Major use and effects

Aladdin·H® incorporates magical oriental therapies including Acupuncture, Thermotherapy, Magnetic therapy, Promotion of vital Energy.

It plays tremendous roles to prevent and treat diseases.

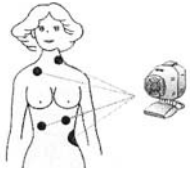
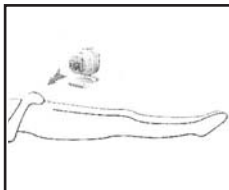
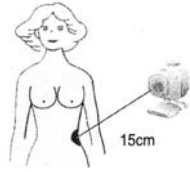
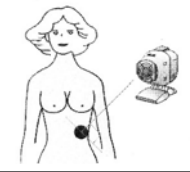

Aladdin·H® maximizes efficacy of other therapies including medication

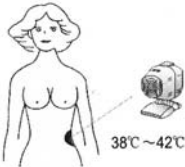
Examples of symptoms to apply Aladdin·H® to with medication

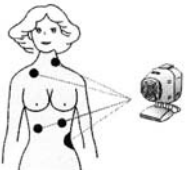
| | |
|-------------------|---|
| Surgery | Soft tissue injury, Burn, Chilblain, Arthritis, Facial paralysis, Neuralgia, Hemorrhoids, Disc... |
| Internal Medicine | Bronchial asthma, Gastric & Duodenal ulcer, Enteritis, Hepatitis, Cholecystitis, Angia pectoris, Diabetes, Hyper-tension... |
| Pediatric | Pneumonia, Dysentery, Nocturia, Growth, Retardation... |
| OB & GY | Menorrhagia, Menstrual trouble, Leucorrhea, White, Endometritis, Mastitis... |
| Dermatology | Eczema, Acne, Skin ulcer, Herpes joster, Dermatitis (Neural, Allergic)... |
| Urology | Cystitis, Prostatitis, Impotence, Incontinentia urine.... |
| E.N.T | Rhinitis, Auris media, Tinnitus, Laryngitis.... |
| Dentistry | Pericoronitis, Toothache, Post-operative trouble... |
| Neuro-Psychiatry | Neuralsthenia, Insomnia, Melancholia, Schizophrenia, Neuro Gastitis..... |

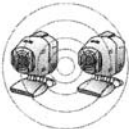
Direction for use


① How to cure

| | |
|--|--|
|  | <p>① Set healing points (origin of illnesses, painful area, and major acupuncture points) depending on the patient's symptom and age. (1-2 points for acute illnesses, 2-4 points for chronic illnesses)</p> |
|  | <p>② Expose the healing point completely, in order to facilitate the penetration of human resonance wave thereby maximizing the effects.</p> |
|  | <p>③ The distance between the healing point and Aladdin-H should be determined by the patients to have a comfortable warm feeling. < The basis healing distance > TR-300 = 15cm, The other models = 25cm from safety wire net.</p> |
|  | <p>④ Vertically place the center of the disk of Aladdin-H and the point lamp so that the wave would well penetrate in the organs of the human body.</p> |
| <p>30-60mins./1day</p>  | <p>⑤ Cure the point for 30-60 minutes per day or every two days. If the symptoms are severe, one hour up to twice per day would be possible.</p> |

| | |
|---|---|
|  | <p>⑥ The temperature of the healing point should be 38-42 °C warming the skin pale pink and the patient have a comfortable warm feeling</p> |
|---|---|

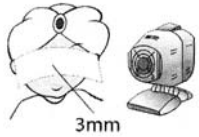
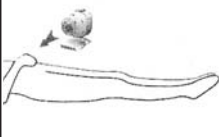



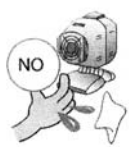
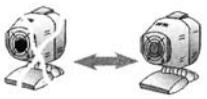
| | |
|---|--|
|  | <p>⑦ When healing multiple points (origin of illnesses, painful area, and major acupuncture points) simultaneously or orderly, increase the healing time; up to 120 minutes per day for the first 10 days of healing</p> |
|---|--|

| | |
|---|--|
|  | <p>⑧ The period of cure depends on symptoms. In case of chronic illnesses, discontinue for 5-7 days after curing 7-10 days and resume repeatedly. Even though the symptoms seemingly disappear, do not immediately discontinue. (Discontinue the cure for 3-7 days and resume the cure in order to prevent reoccurrence of the symptom.)</p> |
|---|--|

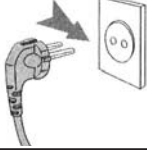
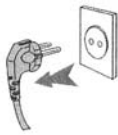
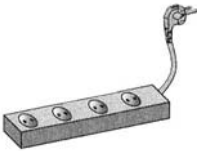


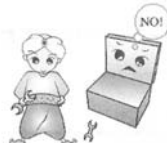

| | |
|---|--|
|  | <p>⑨ Chronic arthritis and neuralgia are likely to accompany some pain. However, this is the process that those chronic diseases run an acute course, and therefore it enables the fundamental cure.</p> |
|---|--|



2 Attention

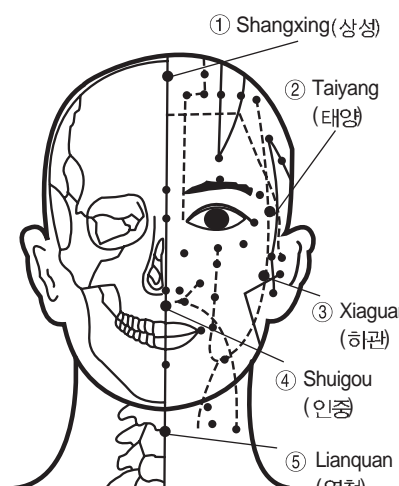
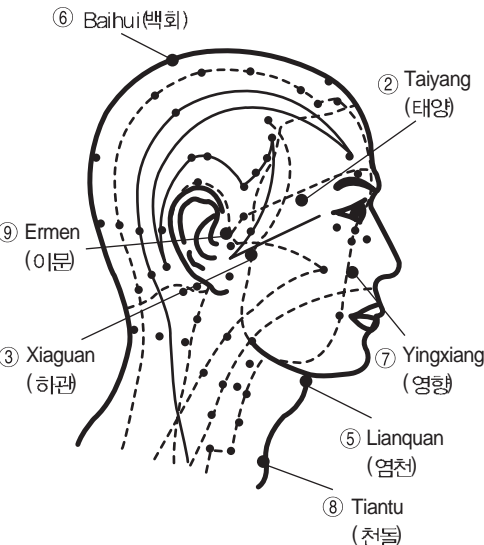
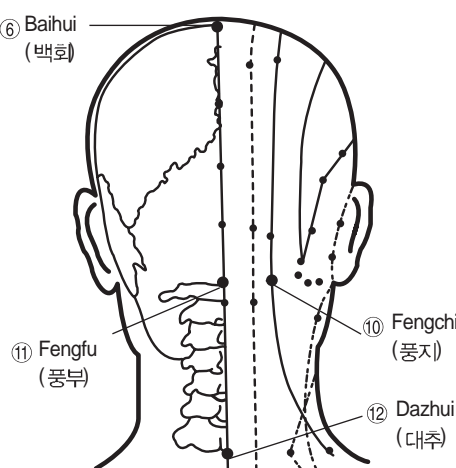
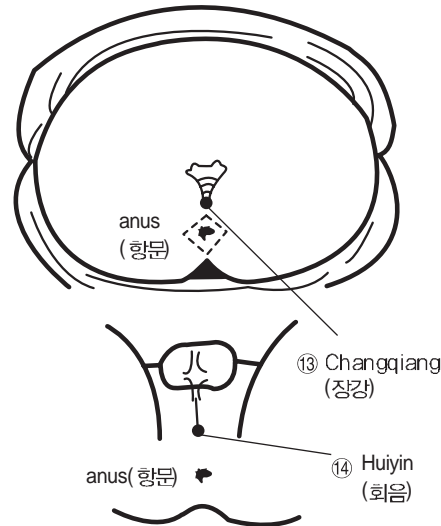
| | |
|--|---|
|  <p>3mm</p> | <p>① When healing on a face or eyes, be sure to protect eye area with cloth which has 3mm or more thickness</p> |
|  | <p>② Do not pose uncomfortably to set the healing point. Adjust Aladdin-H or the pose to feel comfortable points for chronic illnesses</p> |
|  | <p>③ When the patients feel uncomfortable, temporarily stop the cure and adjust the distance or time.</p> |
|  <p>OVER 40cm</p> | <p>④ Do not stay close to Aladdin-H. Maintain the minimum distance where two point lamps (in red) converge.(Especially, a child and a burn patient must be maintained distance over 40cm.)</p> |
|  | <p>⑤ Consult the doctor before Aladdin-H therapy for the following diseases;</p> <ul style="list-style-type: none">- High fever- Obstructive vasculitis- Womb area during pregnancy |
|  | <p>⑥ Do not touch or use wet cloth on the disk or disk area in order to prevent malfunctioning.</p> |
|  | <p>⑦ If the disk breaks or is discolored (originally black becomes brown and gold becomes grey), consult the customer service and replace the disk immediately.</p> |

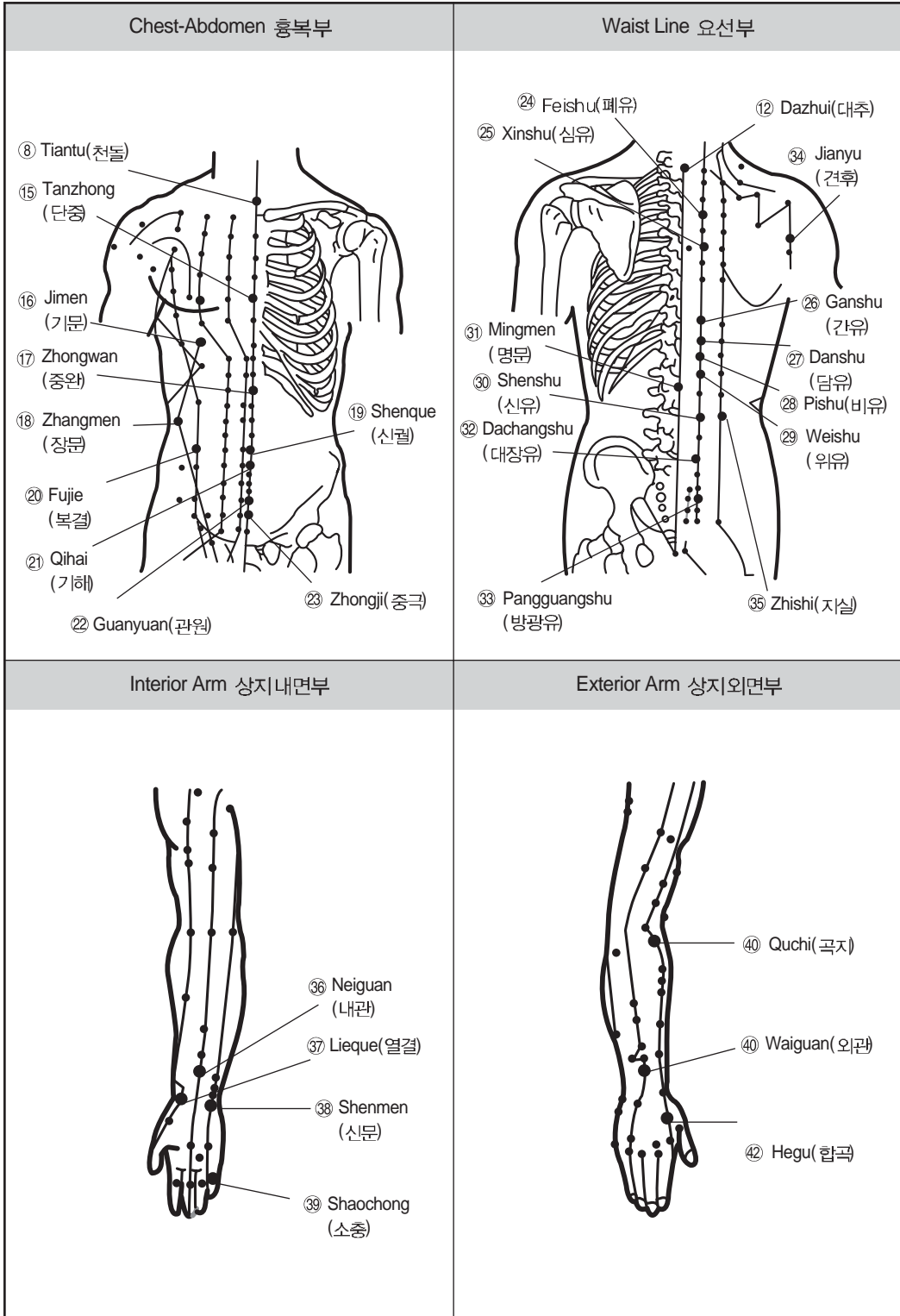
③ Warning

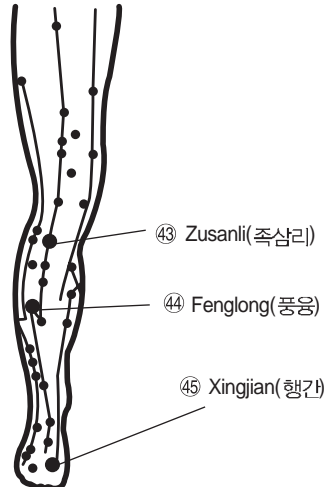
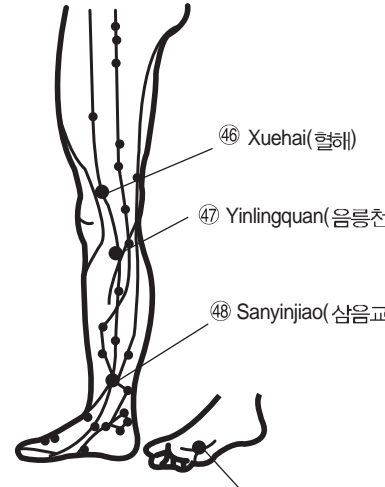
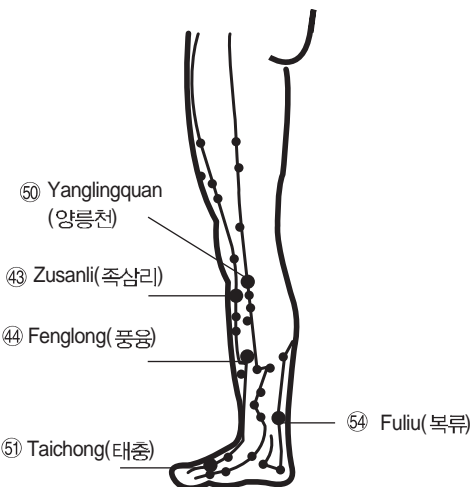
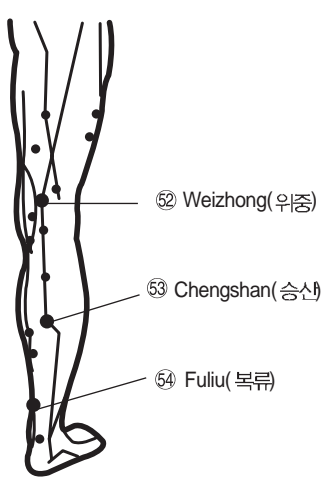
| | |
|---|--|
|  | <p>① Plug the power cord into the electric outlet properly.</p> |
|  | <p>② Do not disconnect the plug when the system is in operation. This could cause a malfunction and reduce the system lifetime.</p> |
|  | <p>③ When an extension cord is necessary, use a product conforming to the following: a proper voltage, length less than 5m. Do not twist or bend the cord, and do not place heavy items on the cable to avoid fire and electric shock.</p> |
|  | <p>④ Read 'Direction for use' carefully before use.</p> |
|  | <p>⑤ Keep out of children.</p> |
|  | <p>⑥ Do not tamper or alter. In case of malfunctioning, consult the customer service</p> |
|  | <p>⑦ Do not touch the plug with wet hands. There is a danger of electric shock.</p> |

4) Healing Point

1) The pertinent number and primary acupoints for each region

| Anterio Head 전두부 | Lateral Head 측두부 |
|---|---|
|  <p>① Shangxing(상성) ② Taiyang (태양) ③ Xiaguan (하관) ④ Shuigou (인중) ⑤ Lianquan (염천)</p> |  <p>② Taiyang (태양) ③ Xiaguan (하관) ⑥ Baihui(백회) ⑦ Yingxiang (영향) ⑧ Tiantu (천돌) ⑨ Ermen (이문)</p> |
| Posterio Head 후두부 | Peri-Anus 항문주위부 |
|  <p>⑥ Baihui (백회) ⑩ Fengchi (풍지) ⑪ Fengfu (풍부) ⑫ Dazhui (대추)</p> |  <p>anus (항문) ⑬ Changqiang (장강) ⑭ Huiyin (회음) anus (항문)</p> |



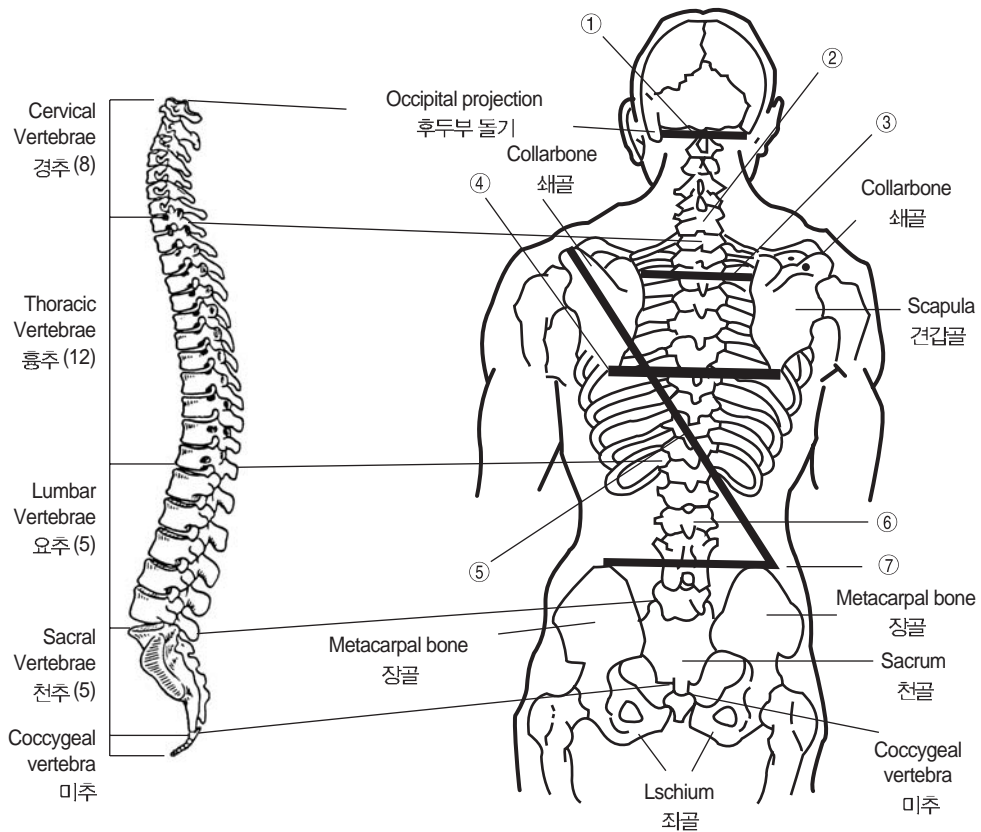
| Facade Lower Extremity 하지전면부 | Interior Lower Extremity 하지내면부 |
|--|---|
|  <p>④③ Zusanli(족삼리)</p> <p>④④ Fenglong(풍옹)</p> <p>④⑤ Xingjian(행간)</p> |  <p>④⑥ Xuehai(혈해)</p> <p>④⑦ Yinlingquan(음릉천)</p> <p>④⑧ Sanyinjiao(삼음교)</p> <p>④⑨ Yongquan(응천)</p> |
| Exterior Lower Extremity 하지측면부 | Posterior Lower Extremity 하지후면부 |
|  <p>⑤① Yanglingquan(양릉천)</p> <p>④③ Zusanli(족삼리)</p> <p>④④ Fenglong(풍옹)</p> <p>⑤④ Fuliu(복류)</p> <p>⑤① Taichong(태충)</p> |  <p>⑤② Weizhong(위중)</p> <p>⑤③ Chengshan(승산)</p> <p>⑤④ Fuliu(복류)</p> |

2) Location of the primary acupoints for each region

| REGION | NO. | POINT | LOCATION |
|---------------------------|-----|----------------------|--|
| Anterio Head 전두부 | ① | Shangxing 상성(上星)Du23 | slightly superior to the lateral/vertical center of the forehead on the anterior |
| | ② | Taiyang 태양(太陽)Ex-HN5 | head directly superior the lateral end of the eyebrow |
| | ③ | Xiaguan 하관(下關)S7 | anterior to the ear in the depression formed when the mouth is opened |
| | ④ | Shuigou 인중(人中)Du26 | midway between the nose and mouth |
| | ⑤ | Lianquan 엄천(廉泉)Ren23 | posterior to the chin |
| Lateral Head 측두부 | ⑥ | Baihui 백회(百會)Du20 | center of the crown of the head |
| | ⑦ | Yingxiang 영향(迎香)LI20 | lateral to the nose |
| | ⑧ | Tiantu 천돌(天突)Ren22 | the projected part of the neck |
| Posterior Head 후두부 | ⑨ | Ermen 이문(耳門)SJ21 | directly anterior to the ear |
| | ⑩ | Fengchi 풍지(風池)G20 | 3cm to the left/right of CV1 in the occipital region |
| | ⑪ | Fengfu 풍부(風府)Du16 | center of CV1 in the occipital region |
| | ⑫ | Dazhui 대추(大椎)Du14 | between CV7 and TV1 |
| Peri-Anus 항문주위부 | ⑬ | Changqiang 장강(長強)Du1 | between the anus and pubic bone |
| | ⑭ | Huiyin 회음(會陰)Ren1 | between the anus and genitals |
| Chest - Abdomen 흉복부 | ⑮ | Tanzhong 단중(檀中)Ren17 | midway between the nipples |
| | ⑯ | Jimen 기문(期門)Sp11 | slightly posterior to the nipples between the 9th~10th rib |
| | ⑰ | Zhongwan 중완(中脘)Ren12 | midway between depression in the chest and navel line |
| | ⑱ | Zhangmen 장문(章門)Liv13 | the flank |
| | ⑲ | Shenque 신궐(神臑)Ren8 | lateral to the navel |
| | ⑳ | Fujie 복결(腹結)Sp14 | posterior to the nipple and lateral to the navel |
| | ㉑ | Qihai 기해(氣海)Ren6 | posterior to the navel |
| | ㉒ | Guanyuan 관원(關元)Ren4 | midway between the navel and sacrum |
| | ㉓ | Zhongji 중극(中極)Ren3 | midway between acupoint 22 and sacrum |
| Waist Line 요선부 | ㉔ | Feishu 폐유(肺俞)B13 | 3cm lateral to TV3, 4 |
| | ㉕ | Xinshu 심유(心俞)B15 | 3cm lateral to TV5, 6 |
| | ㉖ | Ganshu 간유(肝俞)B18 | 3cm lateral to TV9, 10 |
| | ㉗ | Danshu 담유(膽俞)B19 | 3cm lateral to TV10, 11 |
| | ㉘ | Pishu 비유(脾俞)B20 | 3cm lateral to TV11, 12 |
| | ㉙ | Weishu 위유(胃俞)B21 | 3cm lateral to TV12 and LV1 |

| REGION | NO. | POINT | LOCATION |
|---------------------------------------|-----|--------------------------|--|
| Waist Line 요선부 | 30 | Shenshu 신유(腎俞)B23 | 3cm lateral to LV2, 3 |
| | 31 | Mingmen 명문(命門) Du4 | LV2, 3 on the governing vessel meridian 3cm lateral to LV4, 5 |
| | 32 | Dachangshu 대장유(大腸俞)B25 | 3cm lateral to LV4, 5 |
| | 33 | Panguangshu 방광유(膀胱俞)B28 | center of junction between LV5 and the sacrum |
| | 34 | Jiayu 견후(肩髃)L15 | arm/shoulder joint |
| | 35 | Zhishi 지실(至室)B52 | 3cm lateral to acupoint 30 |
| Interior Arm 상지내면부 | 36 | Neiguan 내관(內關)P6 | superior to the wrist crease on the median |
| | 37 | Lieque 열결(列缺)L7 | the wrist crease and back of the thumb |
| | 38 | Shenmen 신문(神門)H7 | the juncture where the wrist crease meets the pinky finger |
| | 39 | Shaohong 소홍(小衝)H9 | pinky finger |
| Exterior Arm 상지외면부 | 40 | Quchi 곡지(曲池)L11 | superior to the elbow crease on the exterior arm |
| | 41 | Waiguan 외관(外關)SJ5 | superior to the wrist on the exterior arm |
| | 42 | Hegu 합곡(合谷)L4 | between the 1st and 2nd fingers |
| Facade Lower Extremity 하지전면부 | 43 | Zusanli 족삼리(足三里)S36 | between the projection on the shinbone and tibia |
| | 44 | Fenglong 풍룡(豐隆)S40 | midway between the ankle and kneecap on the exterior lower extremity |
| | 45 | Xingjian 행간(行間)Liv2 | between the 1st and 2nd toes |
| Interior Lower Extremity 하지내면부 | 46 | Xuehai 혈해(血海)Sp10 | the muscle protruding on the interior part of the knee |
| | 47 | Yinlingquan 음릉천(陰陵泉)Sp9 | posterior to the kneecap on the interior lower extremity |
| | 48 | Sanyinjiao 삼음교(三陰交)Sp6 | superior to the inner anklebone |
| | 49 | Yongquan 용천(湧泉)K1 | at the base of the arch on the foot/sole |
| Exterior Lower Extremity 하지측면부 | 50 | Yanglingquan 양릉천(陽陵泉)G34 | posterior to the kneecap on the exterior lower extremity |
| | 51 | Taichong 태충(太沖)Liv3 | in the crevice between the 1st and 2nd toes |
| Posterior Lower Extremity 하지후면부 | 52 | Weizhong 위중(委中)B40 | midway point of the knee crease on the posterior lower extremity |
| | 53 | Chengshan 승산(承山)B57 | center of the lower end of the gastrocnemius muscle on the posterior lower extremity |
| | 54 | Fuliu 복류(腓溜)K7 | superior to the Achilles tendon on the posterior lower extremity |

Locating parts of the vertebrae on the spine



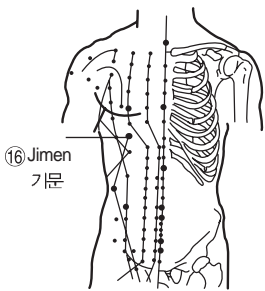

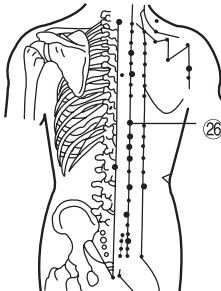
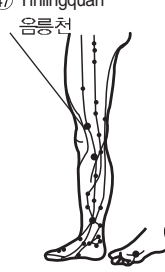
- ① Cervical Vertebrae(CV) 1 경추 1번: the line that extends from the base of the occipital projection
- ② Cervical Vertebrae(CV) 7 경추 7번: the most protruded part of the cervical vertebrae
- ③ Thoracic Vertebrae(TV) 3 흉추 3번: the part that connects horizontally to the interior of the scapula
- ④ Thoracic Vertebrae(TV) 7 흉추 7번: the line connected to the base of the scapula
- ⑤ Thoracic Vertebrae(TV) 11 흉추 11번: the intersection where the metacarpal bone is diagonally connected to the collarbone
- ⑥ Lumbar Vertebrae(LV) 3 요추 3번: directly posterior to the navel
- ⑦ Lumbar Vertebrae(LV) 4 요추 4번: the part that connects to the vertex of the metacarpal bone

3) Representative clinical example of drug treatment and combination

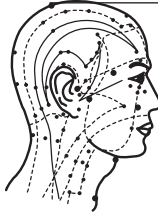
Aladdin-H® can be used on a wide range of illnesses. This is an introduction of the representative clinical applications currently used. In treating illnesses, deciding on an accurate healing point is extremely important. Depending on the symptoms, the duration of treatment can be shortened effectively through drug therapy and concurrent use. Main healing points like the affected area or pain point are generally cured directly but depending on the disease, healing points based on acupoints are cured concurrently or successively, thereby maximizing the effect. Through basic or clinical application on anticipated healing points, we hope to see you overcome the limitations of previous treatments

| Clinical example | |
|--|---|
| 24) Hepatitis, Liver Cirrhosis 간염, 간경화 | |
| M A I N P O I N T | <p>①⑥ Jimen 기문</p> <p>②⑥ Ganshu 간유</p> |
| S U P P O R T P O I N T | <p>④③ Zusanli 족삼리</p> <p>④⑦ Yinlingquan 음릉천</p> |

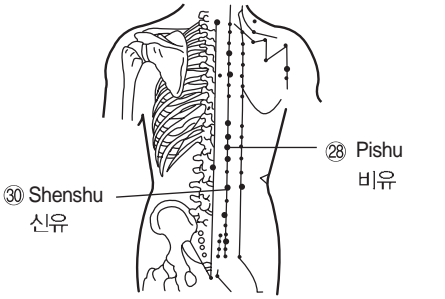
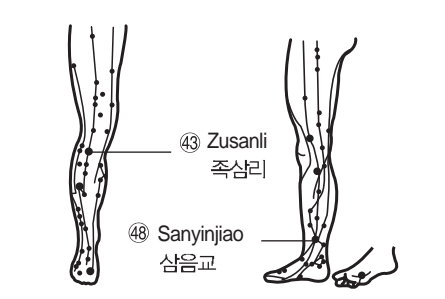
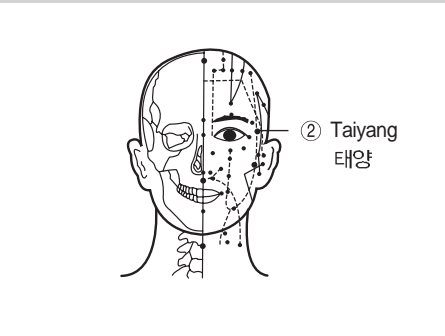
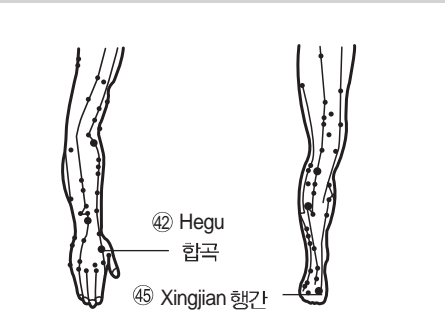
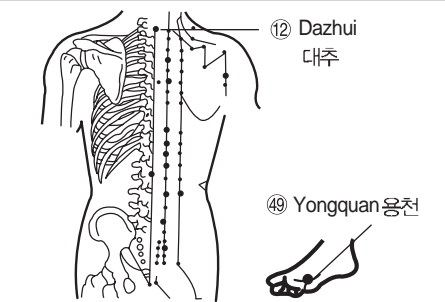
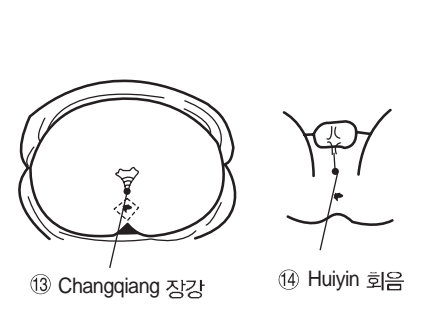
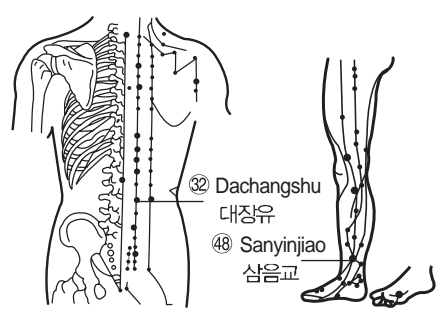
- On the first day, the main point Jimen (30~40min.) and support point Zusanli (15~20min.) are tested and on the second day, the main point Ganshu (30~40min.) and support point Yinlingquan(15~20min.) are treated. This is carried out for a period of 10 days, then following a 5~7day break, treatment is repeated. (Depending on minor ailments or symptoms, support points can be omitted.)
- For serious illnesses, during the first 10 days, treatment can be carried out for 1 hr. up to 2 hrs. at a time in the morning then again at night

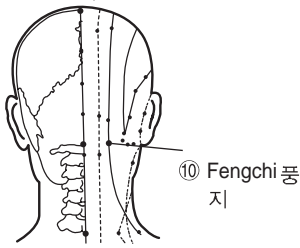

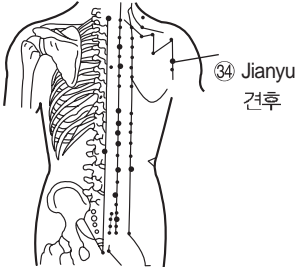
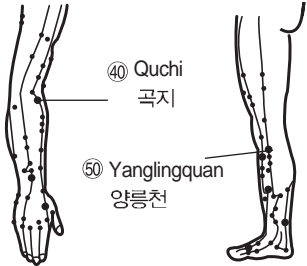
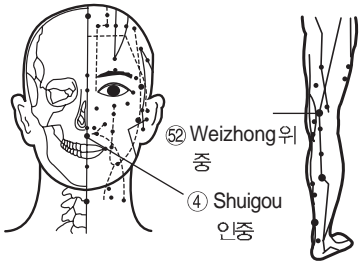
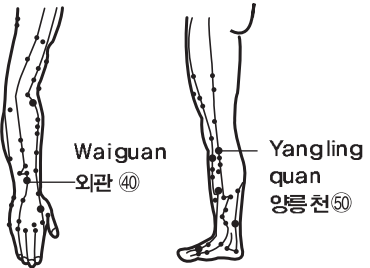
| First day | | Second day | |
|--|--|---|---|
| MAIN POINT | SUPPORT POINT | MAIN POINT | SUPPORT POINT |
|  <p>①⑥ Jimen 기문</p> |  <p>④③ Zusani 족삼리</p> |  <p>②⑥ Ganshu 간유</p> |  <p>④⑦ Yinlingquan 음릉천</p> |
| 40mins.(30mins.) | 20mins(15mins.) | 40mins(30mins.) | 20mins(15mins.) |

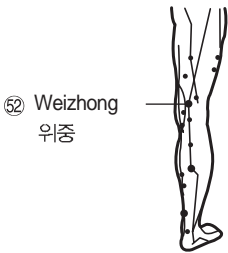
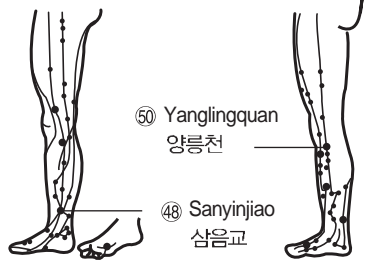
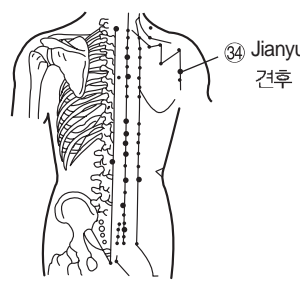
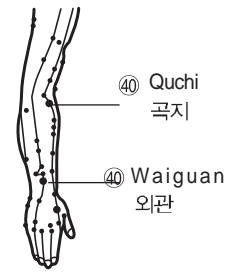
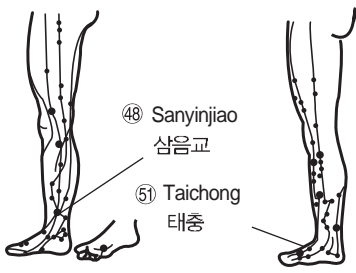
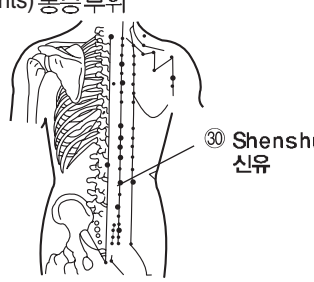
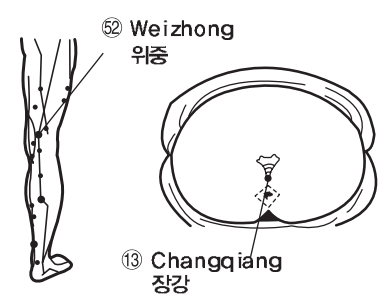
Representative treatment points for drug therapy and combination

| 1) Soft Tissue Injury 연조직 상처(염좌, 좌상, 타박상) | | | |
|---|---------------|--|--|
| M A I N P O I N T | P(Lesions) 환부 | S U P P O R T P O I N T | P(pain points) (환부) |
| 2) Burn 화상 | | | |
| M A I N P O I N T | P(Lesions) 환부 | S U P P O R T P O I N T | (For full body burns)  <p>⑥ Lianquan 백회</p> |

| 3) Sciatica Neuralgia 좌골 신경통 | | | |
|---|---|--|--|
| M A I N P O I N T | P(spinal lesions) 환부 | S U P P O R T P O I N T | <p>⑤② Weizhong 위중</p> <p>⑤③ Chengshan 승산</p> |
| 4) Intercostal Neuralgia 늑간신경통 | | | |
| M A I N P O I N T | P(pain points) 통증부위 | S U P P O R T P O I N T | <p>②⑥ Ganshu 간유</p> <p>③⑥ Neiguan 내관</p> <p>⑤① Taichong 태충</p> |
| 5) Trigeminal Neuralgia 삼차신경통 | | | |
| M A I N P O I N T | <p>② Taeyang 태양</p> <p>③ Xiaguan 하관</p> <p>⑩ Fengchi 풍지</p> | S U P P O R T P O I N T | <p>④⑦ Yinlingquan 음릉천</p> <p>④② Hegu 합곡</p> |
| 6) Facial Paralysis 안면신경마비 | | | |
| M A I N P O I N T | <p>⑥ Baihui 백회</p> <p>③ Xiaguan 하관</p> <p>⑪ Fengfu 풍부</p> | S U P P O R T P O I N T | <p>④② Hegu 합곡</p> |

| 7) Multi-Neuralgia 다발성신경염 | | | |
|---|---|--|---|
| M A I N P O I N T |  <p>③⑩ Shenshu 신유</p> <p>②⑧ Pishu 비유</p> | S U P P O R T P O I N T |  <p>④③ Zusanli 족삼리</p> <p>④⑧ Sanyinjiao 삼음교</p> |
| 8) Migraine 편두통 | | | |
| M A I N P O I N T |  <p>② Taiyang 태양</p> | S U P P O R T P O I N T |  <p>④② Hegu 합곡</p> <p>④⑤ Xingjian 행간</p> |
| 9) Chilblain 동상 | | | |
| M A I N P O I N T | <p>P(spinal lesions) 환부</p> | S U P P O R T P O I N T |  <p>⑫ Dazhui 대추</p> <p>④⑨ Yongquan 용천</p> |
| 10) Hemorrhoids 치질 | | | |
| M A I N P O I N T |  <p>⑬ Changqiang 장강</p> <p>⑭ Huiyin 회음</p> | S U P P O R T P O I N T |  <p>⑳ Dachangshu 대장유</p> <p>④⑧ Sanyinjiao 삼음교</p> |

| 11) Neck Rhermatic Arthritis 목(경부)관절염 | | | |
|---|---|--|--|
| M A I N P O I N T | <p>P(spinal lesions) 환부</p>  <p>⑩ Fengchi 풍지</p> | S U P P O R T P O I N T |  <p>④⑩ Quchi 곡지</p> |
| 12) Omarthritis 견갑관절염 | | | |
| M A I N P O I N T | <p>P(pain points) 통증부위</p>  <p>③④ Jianyu 견후</p> | S U P P O R T P O I N T |  <p>④⑩ Quchi 곡지</p> <p>⑤⑩ Yanglingquan 양릉천</p> |
| 13) Spinal Arthritis 척추관절염 | | | |
| M A I N P O I N T | <p>P(pain points) 통증부위</p> | S U P P O R T P O I N T |  <p>⑤② Weizhong 위중</p> <p>④ Shuigou 인중</p> |
| 14) Coxarthrosis 고(허벅지)관절염 | | | |
| M A I N P O I N T | <p>P(pain points) 통증부위</p> | S U P P O R T P O I N T |  <p>Waiguan 외관 ④⑩</p> <p>Yanglingquan 양릉천 ⑤⑩</p> |

| 15) Knee Joint Arthritis 슬(무릎)관절염 | | | |
|---|---|--|--|
| M A I N P O I N T | <p>P(pain points) 통증부위</p>  <p>⑤② Weizhong 위중</p> | S U P P O R T P O I N T |  <p>⑤① Yanglingquan 양릉천</p> <p>④⑧ Sanyinjiao 삼음교</p> |
| 16) Elbow & Finger Arthritis 주지관절염 | | | |
| M A I N P O I N T | <p>P(pain points) 통증부위</p>  <p>③④ Jianyu 견후</p> | S U P P O R T P O I N T |  <p>④① Quchi 곡지</p> <p>④① Waiguan 외관</p> |
| 17) Foot & Toe Arthritis 족지관절염 | | | |
| M A I N P O I N T | <p>P(pain points) 통증부위</p> | S U P P O R T P O I N T |  <p>④⑧ Sanyinjiao 삼음교</p> <p>⑤① Taichong 태충</p> |
| 18) Disk 디스크 | | | |
| M A I N P O I N T | <p>P(pain points) 통증부위</p>  <p>③① Shenshu 신유</p> | S U P P O R T P O I N T |  <p>⑤② Weizhong 위중</p> <p>①③ Changqiang 장강</p> |

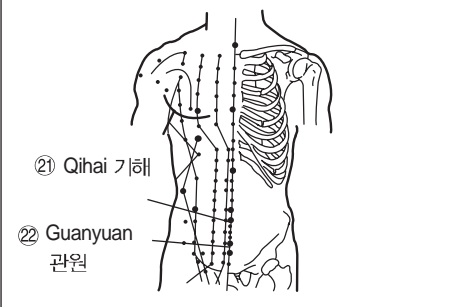
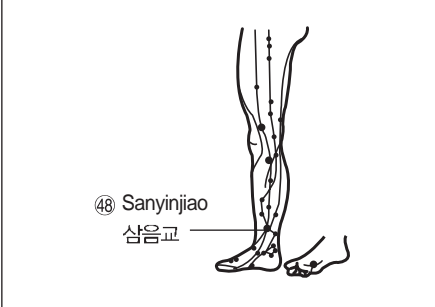
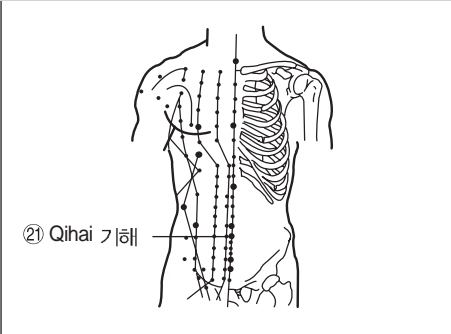
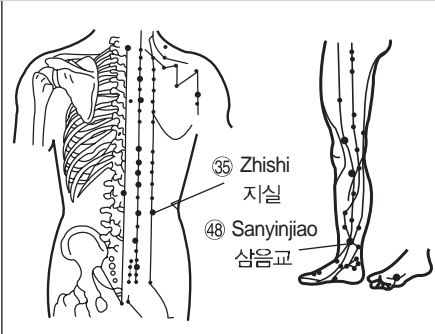
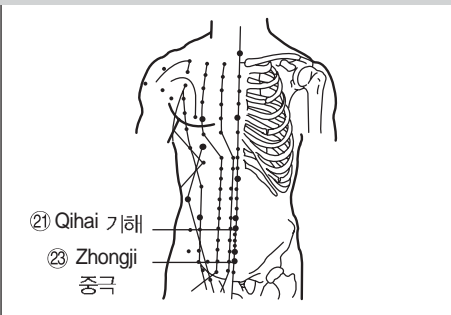
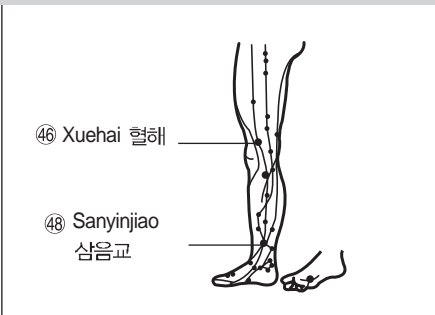
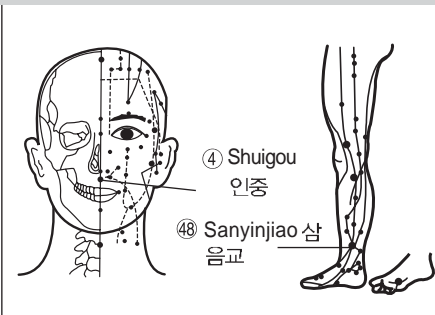
| 19) Bronchitis 기관지염 | | | |
|--|--|--|--|
| M A I N P O I N T | | S U P P O R T P O I N T | |
| 20) Bronchitis Asthma 기관지천식 | | | |
| M A I N P O I N T | | S U P P O R T P O I N T | |
| 21) Gastritis, Duodenal Ulcer 위염, 위 · 십이지장궤양 | | | |
| M A I N P O I N T | | S U P P O R T P O I N T | |
| 22) Enteritis 장염 | | | |
| M A I N P O I N T | | S U P P O R T P O I N T | |

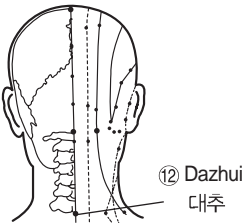
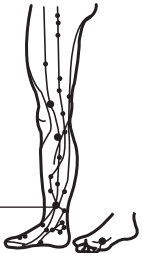
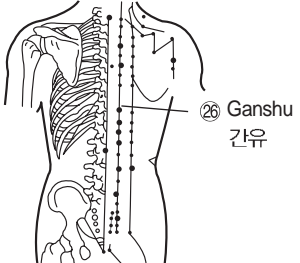
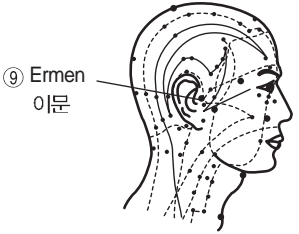
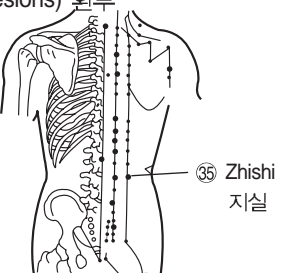
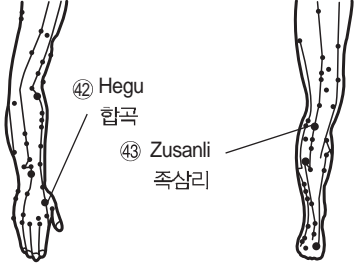
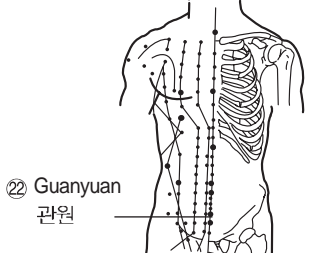
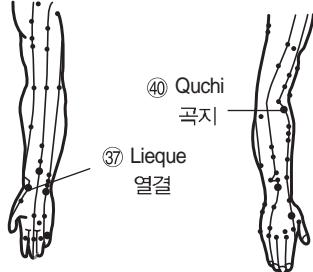
| 23) Constipation 변비 | |
|--|---|
| M A I N P O I N T | <p>32 Dachangshu 대장유</p> <p>21 Qihai 기해</p> |
| S U P P O R T P O I N T | <p>6 Baihui 백회</p> <p>38 Shenmen 신문</p> |
| 24) Hepatitis, Liver Cirrhosis 간염, 간경화 | |
| M A I N P O I N T | <p>26 Ganshu 간유</p> <p>16 Jimen 기문</p> |
| S U P P O R T P O I N T | <p>43 Zusanli 족삼리</p> <p>47 Yinlingquan 음릉천</p> |
| 25) Cholecystitis 담낭염 | |
| M A I N P O I N T | <p>17 Zhongwan 중완</p> <p>27 Danshu 담유</p> |
| S U P P O R T P O I N T | <p>43 Zusanli 족삼리</p> |
| 26) Nephritis 신장염 | |
| M A I N P O I N T | <p>30 Shenshu 신유</p> <p>21 Qihai 기해</p> |
| S U P P O R T P O I N T | <p>50 Yanglingquan 양릉천</p> <p>48 Sanyinjiao 삼음교</p> |

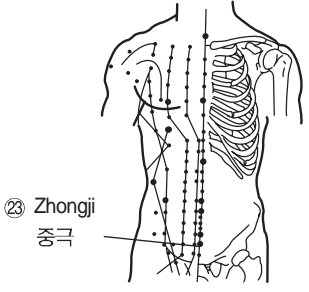
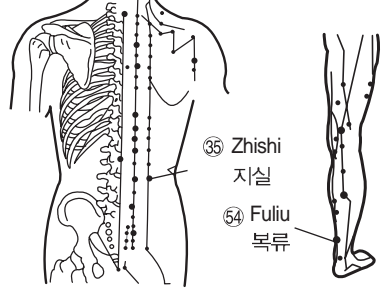
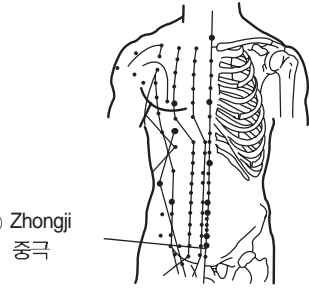
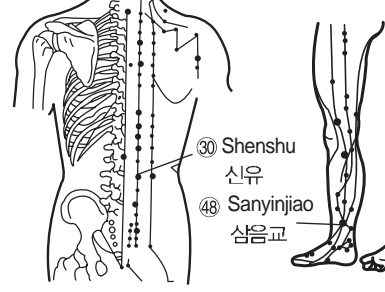
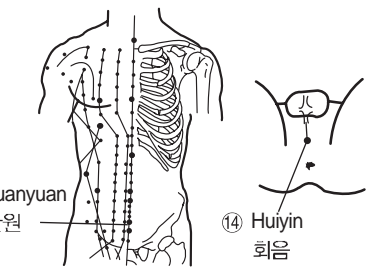
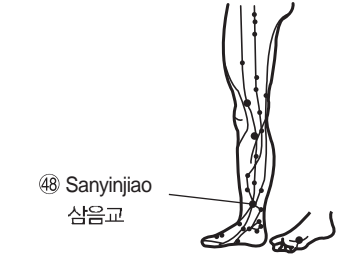
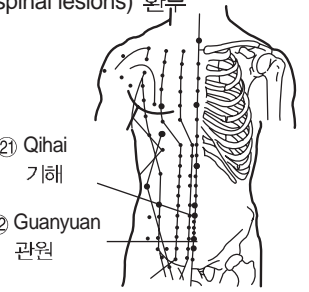
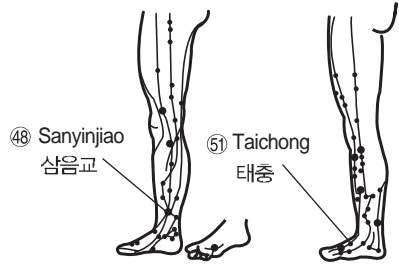
| 27) Cystitis 방광염 | | | |
|---|--|--|--|
| M A I N P O I N T | <p>③③ Pangguangshu 방광유</p> <p>②② Guanyuan 관원</p> | S U P P O R T P O I N T | <p>④⑧ Sanyinjiao 삼음교</p> |
| 28) Diabetes 당뇨병 | | | |
| M A I N P O I N T | <p>①⑥ Jimen 기문 ②⑧ Pishu 비유 (1)</p> <p>③⑩ Shenshu 신유</p> <p>②② Guanyuan 관원 (1)</p> | S U P P O R T P O I N T | <p>⑥ Baihui 백회</p> <p>④③ Zusanli 족삼리</p> |
| 29) Angia Pectoris 협심증 | | | |
| M A I N P O I N T | <p>②⑤ Xinshu 심유</p> | S U P P O R T P O I N T | <p>③⑥ Neiguan 내관</p> <p>③⑧ Shenmen 신문</p> |
| 30) Hyper-Tension 고혈압 | | | |
| M A I N P O I N T | <p>⑥ Baihui 백회</p> <p>⑩ Fengchi 풍치</p> | S U P P O R T P O I N T | <p>②⑥ Ganshu 간유</p> <p>③⑩ Shenshu 신유</p> <p>⑤① Taichong 태충</p> |

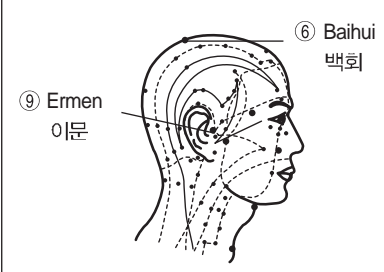
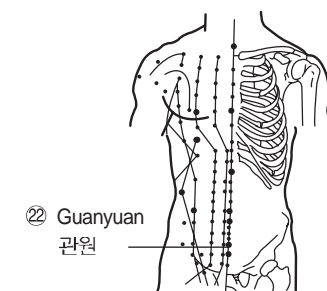
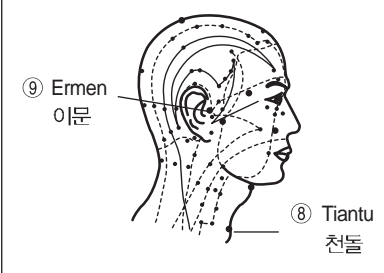
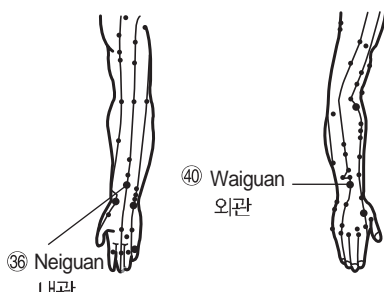
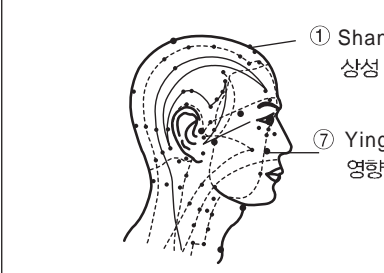
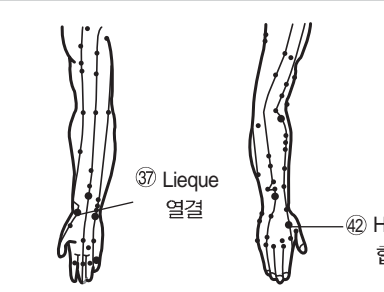
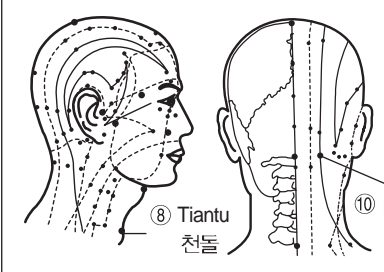
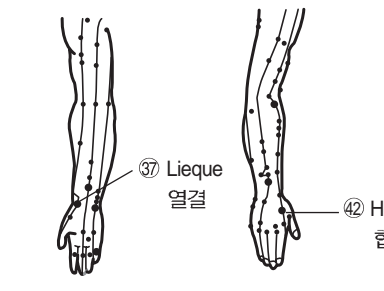
| 31) Hypo-Tension 저혈압 | | | |
|---|--|--|---|
| M A I N P O I N T | <p>⑥ Baihui 백회</p> <p>② Guanyuan 관원</p> | S U P P O R T P O I N T | <p>④③ Zusanli 족삼리</p> <p>④⑧ Sanyinjiao 삼음교</p> |
| 32) Anemia 빈혈 | | | |
| M A I N P O I N T | <p>⑥ Baihui 백회</p> <p>②⑧ Pishu 비유</p> <p>③① Mingmen 명문</p> | S U P P O R T P O I N T | <p>④③ Zusanli 족삼리</p> <p>②② Guanyuan 관원</p> |
| 33) Hypothyroidism 갑상선기능항진 | | | |
| M A I N P O I N T | <p>⑧ Tiantu 천돌</p> | S U P P O R T P O I N T | <p>④② Hegu 합곡</p> <p>④⑧ Sanyinjiao 삼음교</p> |
| 34) Pancreatitis 췌장염 | | | |
| M A I N P O I N T | <p>①⑥ Jimen 기문</p> <p>②⑧ Pishu 비유</p> | S U P P O R T P O I N T | <p>④③ Zusanli 족삼리</p> <p>④⑦ Yinlingquan 음릉천</p> |

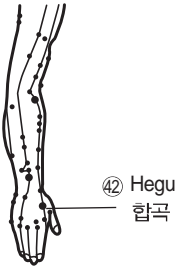
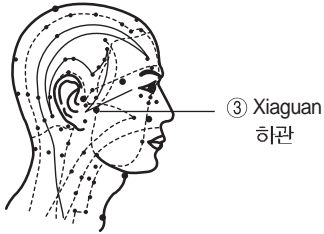
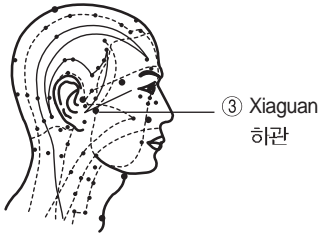


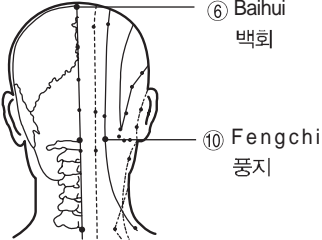
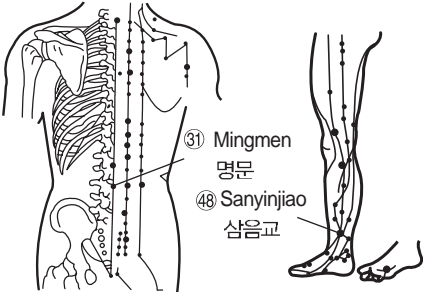
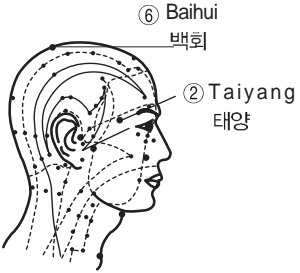

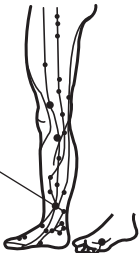
| 35) Infant Pneumonia 소아폐렴 | | | |
|---|--|--|--|
| M A I N P O I N T | <p>24 Feishu 폐유</p> <p>8 Tiantu 천돌</p> | S U P P O R T P O I N T | <p>42 Hegu 합곡</p> |
| 36) Infant Dispepsia 소아소화불량 | | | |
| M A I N P O I N T | <p>19 Shenque 신궐</p> | S U P P O R T P O I N T | <p>43 Zusanli 족삼리</p> <p>42 Hegu 합곡</p> |
| 37) Infant Dysentery 소아하리(설사) | | | |
| M A I N P O I N T | <p>17 Zhongwan 중완</p> <p>19 Shenque 신궐</p> | S U P P O R T P O I N T | <p>43 Zusanli 족삼리</p> |
| 38) Infant Nocturia 소아야뇨증 | | | |
| M A I N P O I N T | <p>22 Guanyuan 관원</p> <p>23 Zhongji 중극</p> | S U P P O R T P O I N T | <p>39 Shaochong 야뇨점</p> <p>48 Sanyinjiao 삼음교</p> |

| 39) Menorrhagia & Menstrual trouble 월경통, 생리불순 | | | |
|---|--|--|--|
| M A I N P O I N T |  <p>②① Qihai 기해 ②② Guanyuan 관원</p> | S U P P O R T P O I N T |  <p>④⑧ Sanyinjiao 삼음교</p> |
| 40) Leucorrhea, Whites 냉증, 대하증 | | | |
| M A I N P O I N T |  <p>②① Qihai 기해</p> | S U P P O R T P O I N T |  <p>③⑤ Zhishi 지실 ④⑧ Sanyinjiao 삼음교</p> |
| 41) Endometritis 자궁내막염 | | | |
| M A I N P O I N T |  <p>②① Qihai 기해 ②③ Zhongji 중극</p> | S U P P O R T P O I N T |  <p>④⑥ Xuehai 혈해 ④⑧ Sanyinjiao 삼음교</p> |
| 42) Mastitis 유선염 | | | |
| M A I N P O I N T | P (spinal lesions) 환부 | S U P P O R T P O I N T |  <p>④ Shuigou 인중 ④⑧ Sanyinjiao 삼음교</p> |

| 43) Eczema, Nuoro-Dermatitis 습진, 신경성피부염 | | | |
|---|---|--|---|
| M A I N P O I N T | P(spinal lesions) 환부  <p>⑫ Dazhui 대추</p> | S U P P O R T P O I N T |  <p>④⑧ Sanyinjiao 삼음교</p> |
| 44) Herpes Zoster 대상포진 | | | |
| M A I N P O I N T | P(spinal lesions) 환부  <p>②⑥ Ganshu 간유</p> | S U P P O R T P O I N T |  <p>⑨ Ermen 이문</p> |
| 45) Alergy Dermatitis 일러르기성 피부염 | | | |
| M A I N P O I N T | P(spinal lesions) 환부  <p>③⑤ Zhishi 지실</p> | S U P P O R T P O I N T |  <p>④② Hegu 합곡 ④③ Zusanli 족삼리</p> |
| 46) Acne 여드름 | | | |
| M A I N P O I N T | P(spinal lesions) 환부 Female (여성)  <p>②② Guanyuan 관원</p> | S U P P O R T P O I N T |  <p>④① Quchi 곡지 ③⑦ Lieque 열결</p> |

| 47) Impotence 임포텐스 | | | |
|---|---|--|--|
| M A I N P O I N T |  <p>㉓ Zhongji 중극</p> | S U P P O R T P O I N T |  <p>㉔ Zhishi 지실 ㉕ Fuliu 복류</p> |
| 48) Incontinentia Urine 요실금 | | | |
| M A I N P O I N T |  <p>㉓ Zhongji 중극</p> | S U P P O R T P O I N T |  <p>㉖ Shenshu 신유 ㉗ Sanyinjiao 삼음교</p> |
| 49) Prostatitis 전립선염 | | | |
| M A I N P O I N T |  <p>㉘ Guanyuan 관원 ㉙ Huiyin 회음</p> | S U P P O R T P O I N T |  <p>㉚ Sanyinjiao 삼음교</p> |
| 50) Testitis & Epididymitis 고환염, 부고환염 | | | |
| M A I N P O I N T | <p>P(spinal lesions) 환부</p>  <p>㉛ Qihai 기해 ㉜ Guanyuan 관원</p> | S U P P O R T P O I N T |  <p>㉝ Sanyinjiao 삼음교 ㉞ Taichong 태충</p> |

| 51) Auris Media 중이염 | | | |
|---|---|--|--|
| M A I N P O I N T |  <p>⑥ Baihui 백회</p> <p>⑨ Ermen 이문</p> | S U P P O R T P O I N T |  <p>②② Guanyuan 관원</p> |
| 52) Tinnitus 이명 | | | |
| M A I N P O I N T |  <p>⑨ Ermen 이문</p> <p>⑧ Tiantu 천돌</p> | S U P P O R T P O I N T |  <p>③⑥ Neiguan 내관</p> <p>④④ Waiguan 외관</p> |
| 53) Rhinitis, Sinusitis 비염 축농증 | | | |
| M A I N P O I N T |  <p>① Shangxing 상성</p> <p>⑦ Yingxiang 영향</p> | S U P P O R T P O I N T |  <p>③⑦ Lieque 열결</p> <p>④② Hegu 합곡</p> |
| 54) Laryngitis 인후염 | | | |
| M A I N P O I N T |  <p>⑧ Tiantu 천돌</p> <p>⑩ Fengchi 풍지</p> | S U P P O R T P O I N T |  <p>③⑦ Lieque 열결</p> <p>④② Hegu 합곡</p> |

| 55) Pericoronitis 치주염 | | | |
|---|--|--|--|
| M A I N P O I N T | P(spinal lesions) 환부 | S U P P O R T P O I N T |  <p>④② Hegu 합곡</p> |
| |  <p>③ Xiaguan 하관</p> | | |
| 56) Toothache 치통 | | | |
| M A I N P O I N T |  <p>③ Xiaguan 하관</p> | S U P P O R T P O I N T |  <p>④② Hegu 합곡</p>  <p>④③ Zusanli 족삼리</p> |
| | | | |
| 57) Neurasthenia 신경쇠약 | | | |
| M A I N P O I N T |  <p>⑥ Baihui 백회</p> <p>⑩ Fengchi 풍치</p> | S U P P O R T P O I N T |  <p>③① Mingmen 명문</p> <p>④⑧ Sanyinjiao 삼음교</p> |
| | | | |
| 58) Insomnia 불면증 | | | |
| M A I N P O I N T |  <p>⑥ Baihui 백회</p> <p>② Taiyang 태양</p> | S U P P O R T P O I N T |  <p>④⑤ Xingjian 행간</p>  <p>④⑧ Sanyinjiao 삼음교</p> |
| | | | |

Sequela of chemotherapy

| 59) Pain 통증 | | | |
|---|--|--|--|
| M A I N P O I N T | P(pain points)통증부위 / P(spinal lesions)환부 | S U P P O R T P O I N T | <p>⑥ Baihui 백회</p> <p>④⑨ Yongquan 용천</p> |
| 60) Vomiting 구토 | | | |
| M A I N P O I N T | <p>⑰ Zhongwan 중완</p> <p>②⑨ Weishu 위유</p> | S U P P O R T P O I N T | <p>⑥ Baihui 백회</p> <p>④③ Zusanli 족삼리</p> |
| 61) Anemia 빈혈 | | | |
| M A I N P O I N T | <p>⑥ Baihui 백회</p> <p>③① Mingmen 명문</p> | S U P P O R T P O I N T | <p>②② Guanyuan 관원</p> |
| 62) Leucopenia 백혈구감소 | | | |
| M A I N P O I N T | <p>⑰ Jimen 기문</p> | S U P P O R T P O I N T | <p>②⑥ Ganshu 간유</p> <p>③① Shenshu 신유</p> |

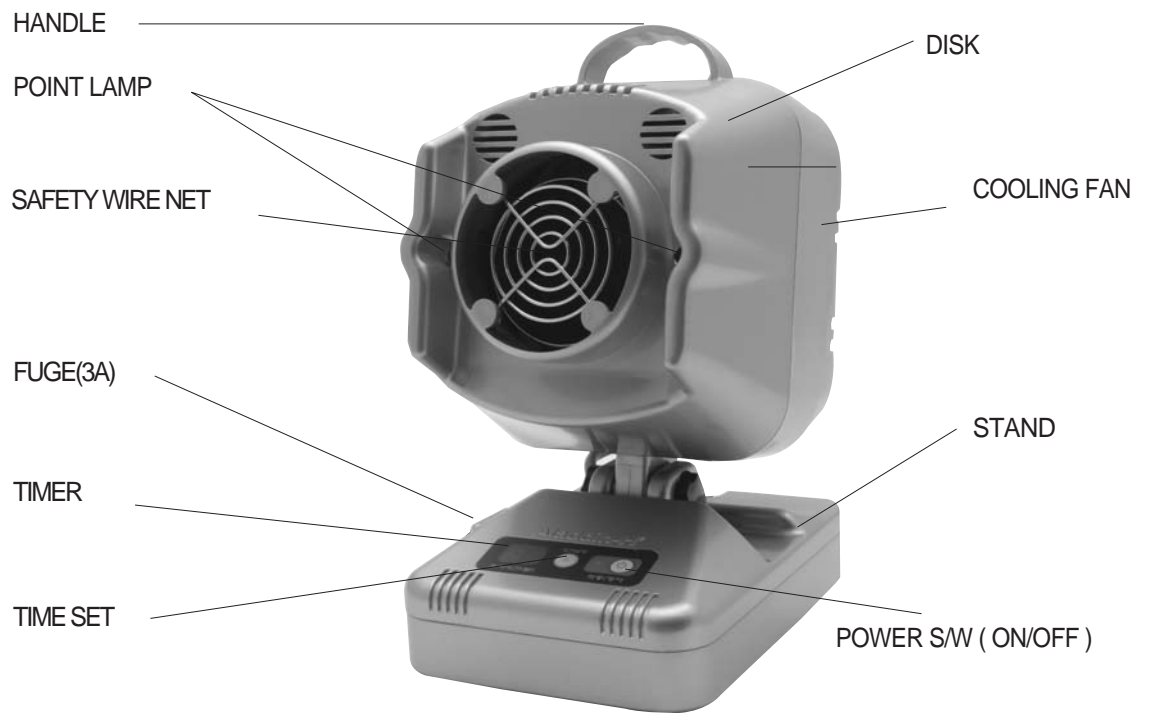
3

Direction for operation

1

TR-300

(1) Design and part names



(2) Direction for operation (TR-300)

- ① Insert the plug to a 220V/110V outlet
- ② Push the POWER S/W; as the operation lamp flickers, it begins to work. This automatically sets time to 30 minutes and operates heating system, cooling fan, point lamp.

ⓄThe junction of left&right point lamp becomes the healing point.
An appropriate distance from the safety net is about 15cm.

- ③ Set the timer; For longer or shorter time, push the TIME SET button multiple times; each time it adds 10 minutes up from 10 to 60 minutes.
- ④ When the time is over, TR-300 system stops and sounds the alarm automatically.

ATTENTION (This is not a malfunction)

If the system is stopped by the on/off button, the cooling fan and point lamps would not stop immediately. Allow 90 more seconds to cool down and protect the disk.

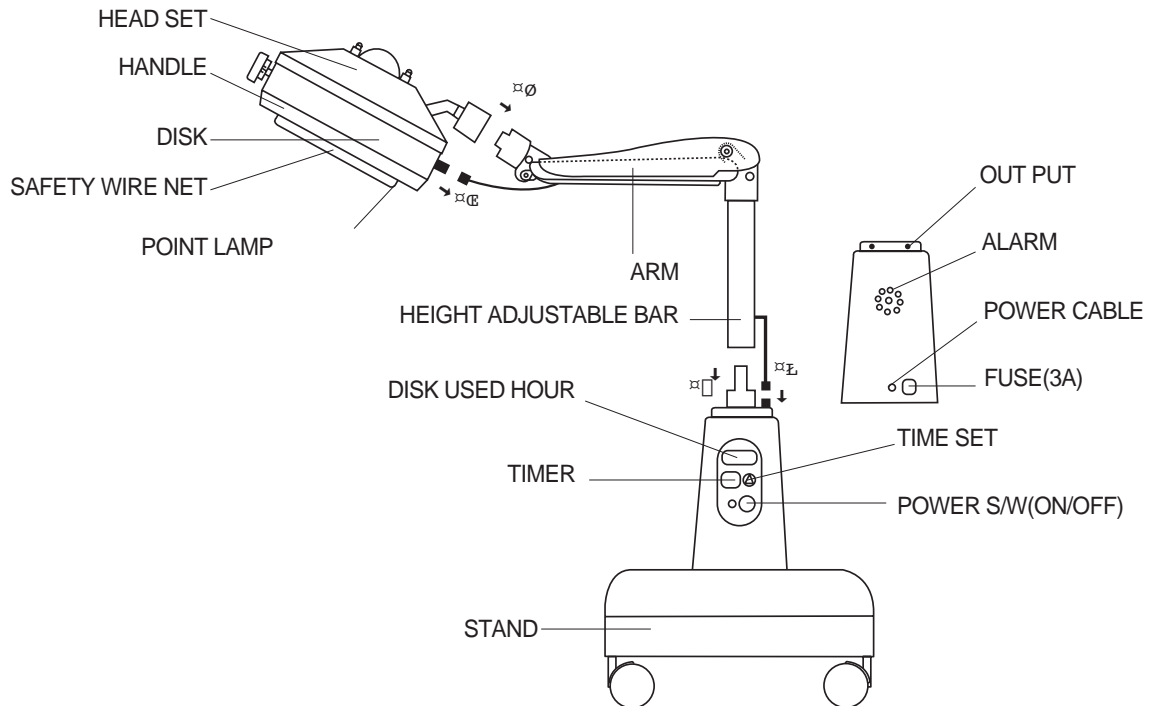
(3) Precautions

- ① Once it is turned on, it takes about 3 mins to preheat.
- ② Before use, read carefully and understand the 'How to cure' (P7-8) and 'Attention/Warning' (P9~10) sections.

(4) Specification

| | |
|----------------------------|--------------------------------|
| Electric Power Source | AC220Volt,110Volt / 50Hz, 60Hz |
| Electric Power Consumption | 200W \pm 10% |
| Disk Temperature | Avg. 320°C(MAX 380°C) |
| Wavelength Range | 2~25 micron |
| Disk Durability | 3,000 hrs. |
| Types of Disk | GOLD · BLACK |
| External Size(W xL xH) | 140 x180 x260mm |

(1) Design and part names



Directions for Assemble

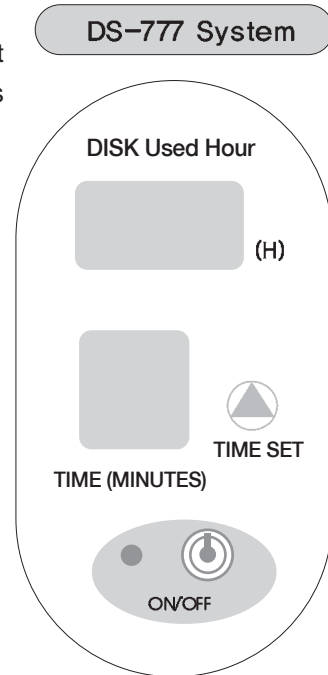
- ① Affix Arm and Height Adjustable Bar onto the main body using a hexagonal wrench.
- ② Link the Connector of the Arm to Output
- ③ Fit the Head Cap to the Arm
- ④ Link the Connector of the Arm to Input of the Head Cap

(2) Direction for operation (DS-777&DS-777S)

- ① Insert the plug to a 220V/110V outlet
- ② Push the POWER S/W; as the operation lamp flickers, it begins to work. This automatically sets time to 30 minutes and operates heating system, point lamp.

The junction of left&right point lamp becomes the healing point.
An appropriate distance from the safety net is about 25cm.

- ③ Set the timer; For longer or shorter time, push the TIME SET button multiple times; each time it adds 10 minutes up from 10 to 60 minutes.
- ④ When the time is over, DS-777 system stops and sounds the alarm automatically.



(3) Precautions

- ① Once it is turned on, it takes about 3 mins to preheat.
- ② Before use, read carefully and understand the 'How to cure' (P7-8) and 'Attention/Warning' (P9-10) sections.
- ③ The DISK USED HOUR indicates the total usage time, contact us to have the disk replaced before it reaches 3,000 hrs.

(4) Specification

| | | |
|----------------------------|--------------------------------|-----------------|
| Electric Power Source | AC220Volt,110Volt / 50Hz, 60Hz | |
| Electric Power Consumption | 200W ±10% | |
| Disk Temperature | Avg. 320°C(MAX 380°C) | |
| Wavelength Range | 2~25 micron | |
| Disk Durability | 3,000 Hrs. | |
| Types of Disk | GOLD · BLACK | |
| External Size(W ×L ×H) | DS-777 | DS-777S |
| | 265 ×310 ×595mm | 320 ×380 ×820mm |

3

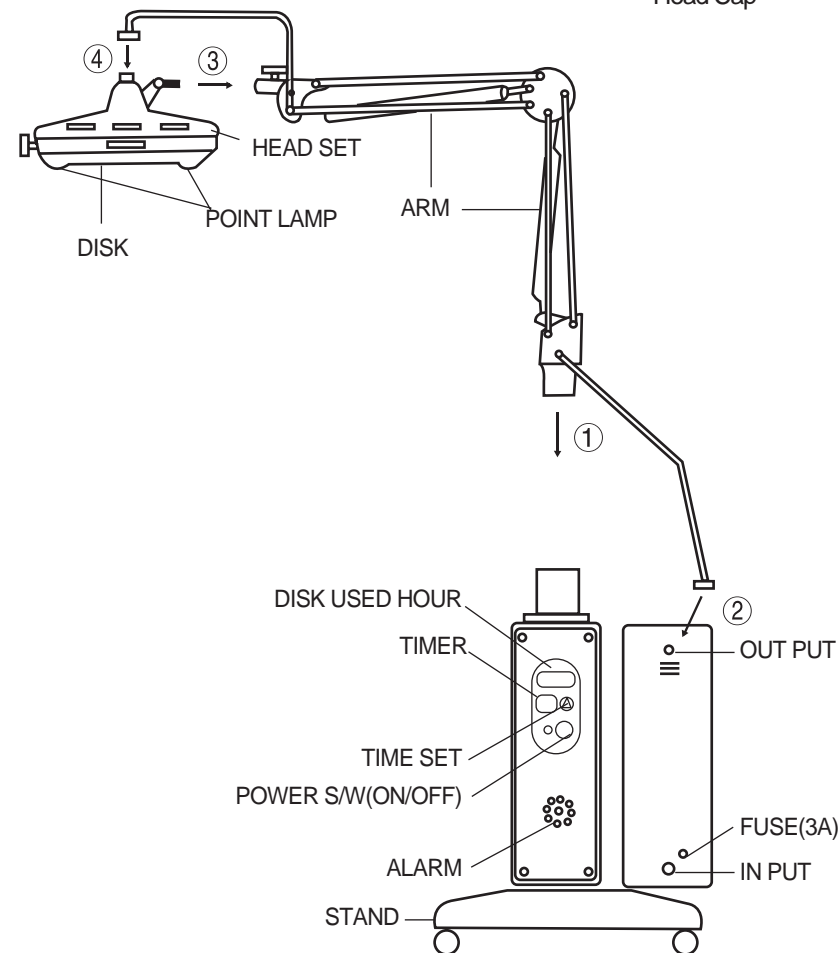
Direction for operation

3 TR-1000& TR-1000S

(1) Design and part names

Directions for Assemble

- ① Affix Arm onto the main body
- ② Link the Connector of the Arm to Output
- ③ Fit the Head Cap to the Arm
- ④ Link the Connector of the Arm to Jack of the Head Cap



(2) Direction for operation(TR-1000&TR-1000S)

- ① Insert the plug to a 220V/110V outlet
- ② Push the POWER S/W; as the operation lamp flickers, it begins to work. This automatically sets time to 30 minutes and operates heating system, point lamp.

The junction of left&right point lamp becomes the healing point.
An appropriate distance from the safety net is about 25cm.

- ③ Set the timer; For longer or shorter time, push the TIME SET button multiple times; each time it adds 10 minutes up from 10 to 60 minutes.
- ④ When the time is over, the TR-1000&TR-1000S system stops and sounds the alarm automatically.

(3) Precautions

- ① Once it is turned on, it takes about 3 mins to preheat.
- ② Before use, read carefully and understand the 'How to cure' (P7-8) and 'Attention/Warning' (P9~10) sections.
- ③ The DISK USED HOUR indicates the total usage time, contact us to have the disk replaced before it reaches 3,000 hrs.

(4) Specification

| | | |
|----------------------------|--------------------------------|-------------------|
| Electric Power Source | AC220Volt,110Volt / 50Hz, 60Hz | |
| Electric Power Consumption | 230W ±10% | |
| Disk Temperature | Avg. 320 °C(MAX 380 °C) | |
| Wavelength Range | 2~25 micron | |
| Disk Durability | 3,000 hrs. | |
| Types of Disk | GOLD · BLACK | |
| External Size(W ×L ×H) | TR-1000 | TR-1000S |
| | 300 ×350 ×900mm | 500 ×520 ×1,280mm |

3

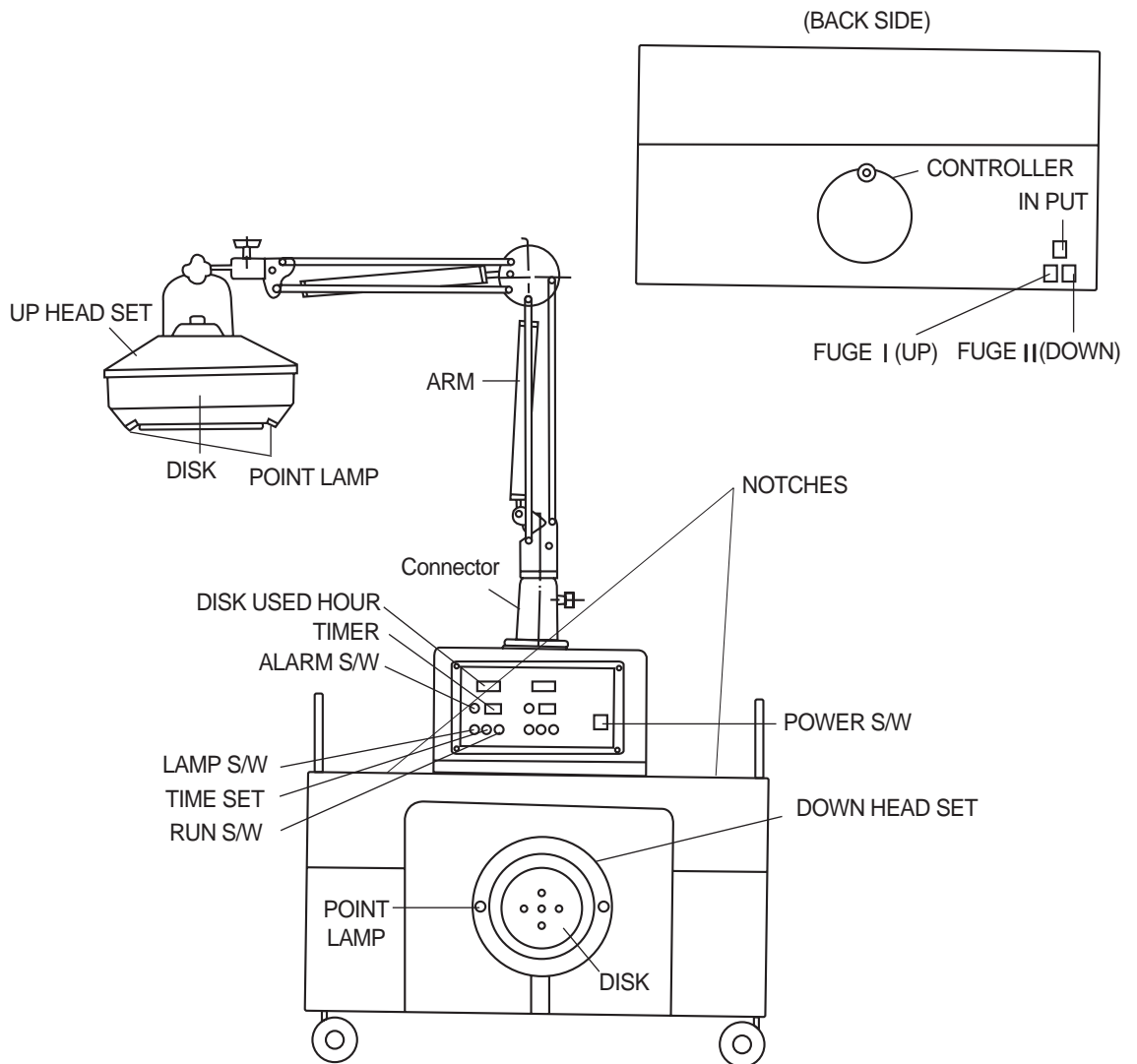
Direction for operation

4

TR-2000

(1) Design and part names

※ Refer to the TR-1000's Directions for Assemble (p38)



(2) Direction for operation (TR-2000)

- ① Insert the plug to a 220V/110V outlet and push the Power button
- ② Push the TIME SET button to set the healing time for both up/down sets.
(Can set healing time from 5 to 60 mins)
- ③ Push the RUN S/W; it operates heating system.

- ⦿ To treat the abdominal region concurrently with the down head, raise both legs onto the left and right NOTCHES.
- ⦿ Push the LAMP S/W to turn on or off the POINT LAMP.
- ⦿ The junction of left&right point lamp becomes the healing point.
An appropriate distance from the safety net is about 25cm.
- ⦿ Setting the alarm will sound the alarm when the healing time has ended.

- ④ Once the preset time has ended the device suspends operation.

(3) Precautions

- ① Once it is turned on, it takes about 3 mins to preheat.
- ② Before use, read carefully and understand the 'How to cure' (P7-8) and 'Attention/Warning' (P9-10) sections.
- ③ The DISK USED HOUR indicates the total usage time, contact us to have the disk replaced before it reaches 3,000 hrs.

(4) Specification

| | |
|----------------------------|--------------------------------|
| Electric Power Source | AC220Volt,110Volt / 50Hz, 60Hz |
| Electric Power Consumption | 230W \pm 10% |
| Disk Temperature | Avg. 320 °C(MAX 380 °C) |
| Wavelength Range | 2~25 micron |
| Disk Durability | 3,000 Hrs. |
| Types of Disk | GOLD · BLACK |
| External Size(W ×L ×H) | 480 ×400 ×854mm |

3

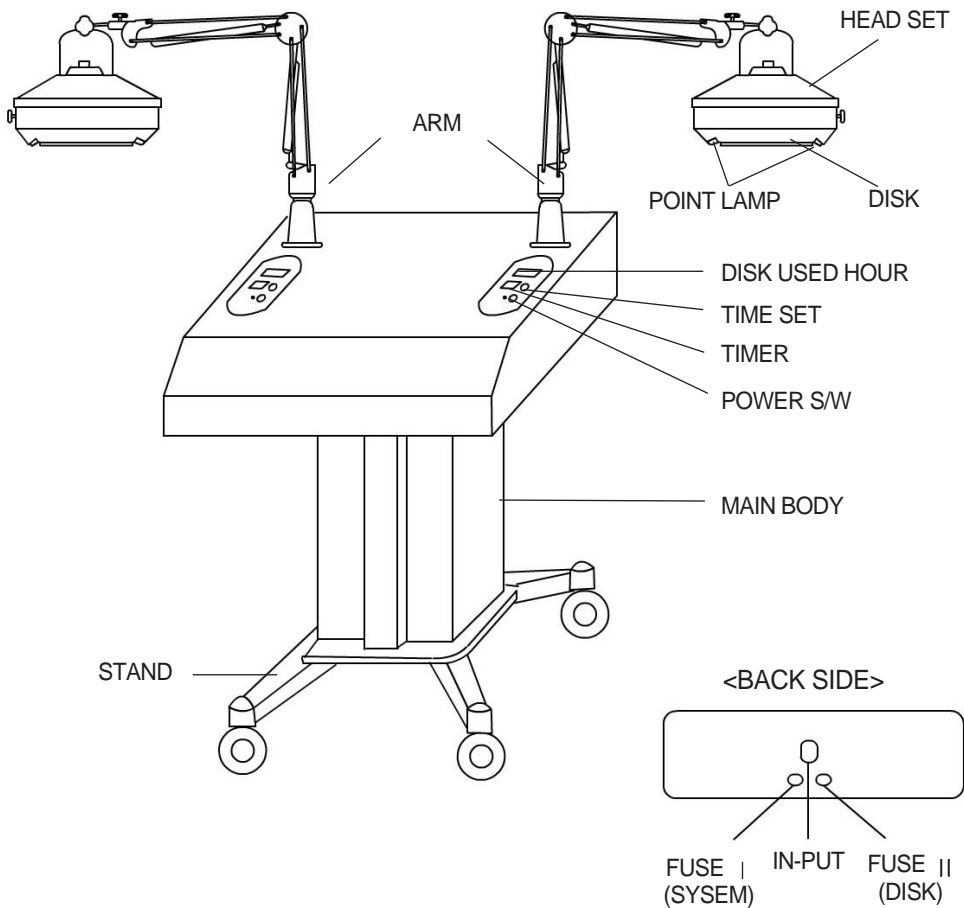
Direction for operation

5

TR-2000D

(1) Design and part names

※ Refer to the TR-1000's Directions for Assemble (p38)



(2) Direction for operation (TR-2000D)

- ① Insert the plug to a 220V/110V outlet.
- ② Push the POWER S/W; as the point lamp on, it begins to work.
- ③ Push the TIME SET button to set the healing time for both left/right sets.
(Can set healing time from 10 to 60 mins)

- ⦿ For effective treatment, can adjust the arm in the reverse direction.
- ⦿ The junction of left&right point lamp becomes the healing point.
An appropriate distance from the safety net is about 25cm.

- ④ When the time is over, the TR-2000D system stops and sounds the alarm automatically.

(3) Precautions

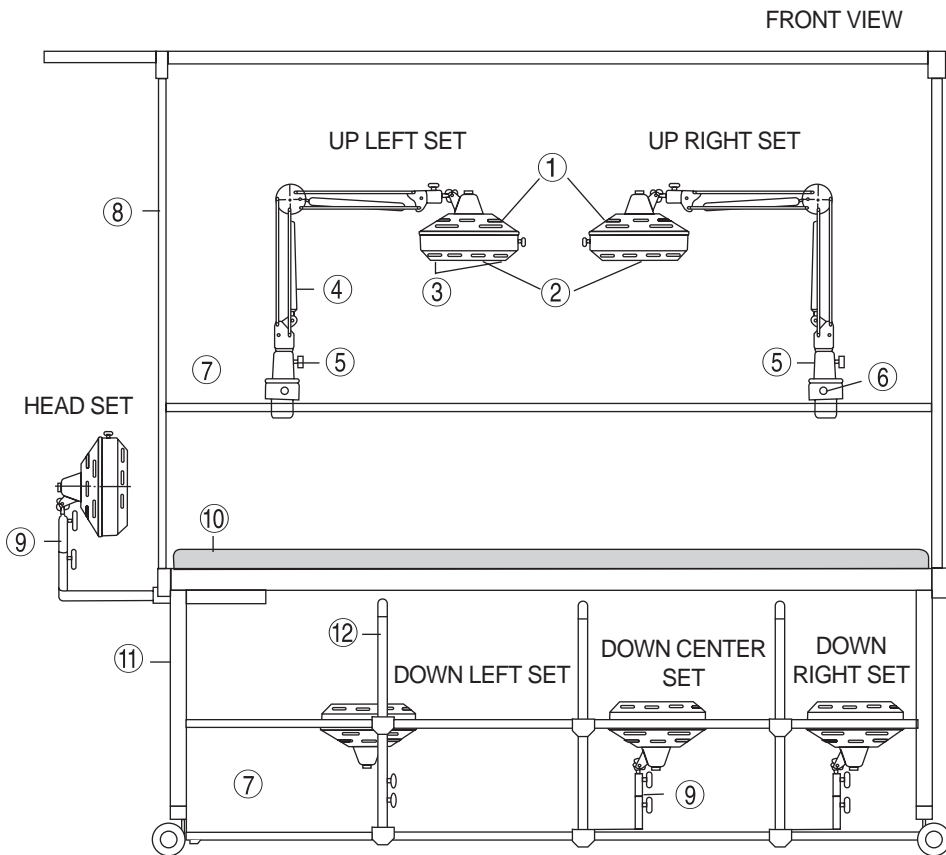
- ① Once it is turned on, it takes about 3 mins to preheat.
- ② Before use, read carefully and understand the 'How to cure' (P7-8) and 'Attention/Warning' (P9~10) sections.
- ③ The DISK USED HOUR indicates the total usage time, contact us to have the disk replaced before it reaches 3,000 hrs.

(4) Specification

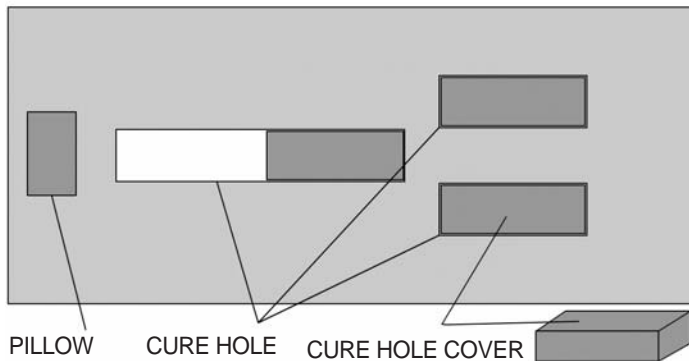
| | |
|--|-----------------------------------|
| Electric Power Source | AC220Volt,110Volt / 50Hz, 60Hz |
| Electric Power Consumption | 230W \pm 10% \times 2 |
| Disk Temperature | Avg. 320°C(MAX 380°C) |
| Wavelength Range | 2~25 micron |
| Disk Durability | 3,000 Hrs. |
| Types of Disk | GOLD · BLACK |
| External Size(W \times L \times H) | 842 \times 630 \times 1,503mm |

Direction for operation

(1) Design and part names

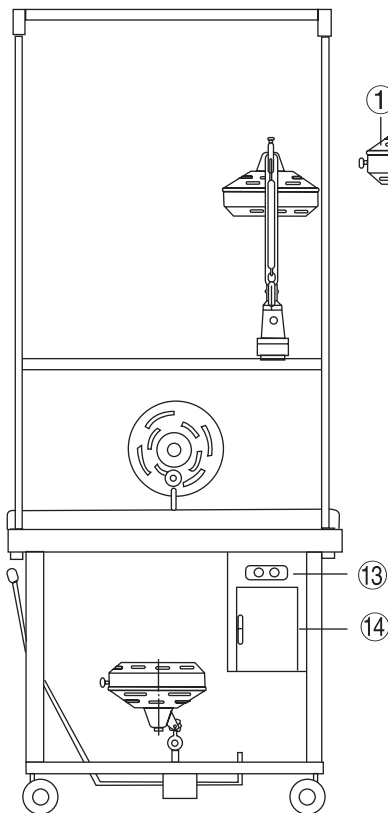


<MATTRESS>

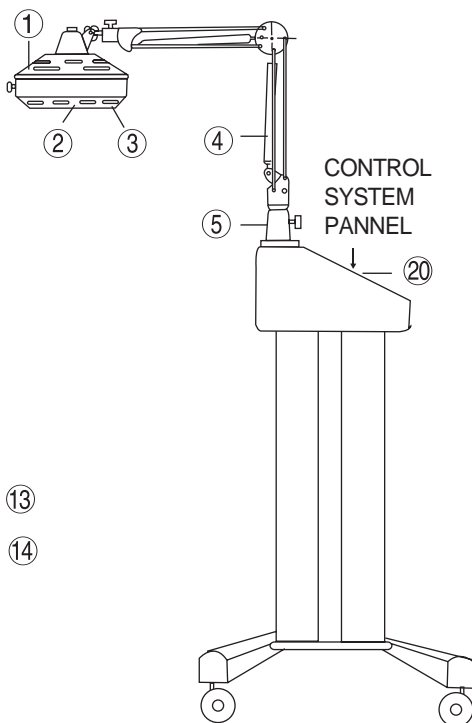


<TR-7000 CONTROL BOX>

SIDE VIEW

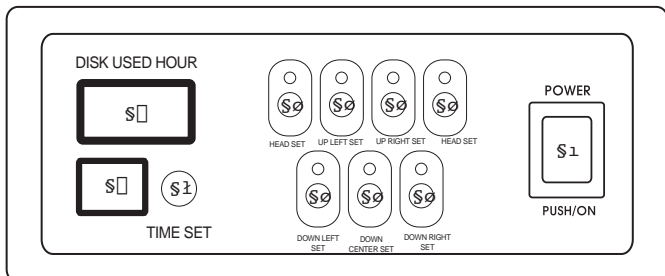


<ASSISTANCE SET>

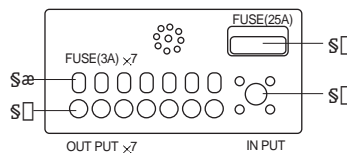


| NO. | PART NAME |
|-----|-----------------------|
| ① | HEAD CAP (7EA) |
| ② | DISK (7EA) |
| ③ | LAMP (7EA) |
| ④ | ARM |
| ⑤ | CONNECTOR |
| ⑥ | OUT-PUT |
| ⑦ | SLIDING BAR |
| ⑧ | CURTAIN HOLDER |
| ⑨ | CONTROL CONNECTOR |
| ⑩ | MATTRESS |
| ⑪ | TABLE |
| ⑫ | SLIDING HANDLE |
| ⑬ | ASSISTANT OUTLET |
| ⑭ | DISTRIBUTING BOARD |
| ⑮ | CONTROL BOX |
| ⑯ | FUSE I (20A) |
| ⑰ | FUSE II(3A)(7EA) |
| ⑱ | IN-PUT |
| ⑲ | OUT-PUT(6EA) |
| ⑳ | CONTROL SYSTEM PANNEL |
| ㉑ | POWER S/W |
| ㉒ | DISK USED HOUR |
| ㉓ | TIMER |
| ㉔ | TIME SET (7EA) |
| ㉕ | RUN S/W (7EA) |

<TR-7000 CONTROL SYSTEM>



BACK SIDE



(2) Direction for operation (TR-7000)

- ① Insert the plug to a 220V/110V outlet and push the Power button.
- ② To use each SET, push each RUN S/W button; as the operation lamp flickers, it begins to work. This automatically sets time to 30 minutes and operates heating system, point lamp.
- ③ Set the timer; For longer or shorter time, push the TIME SET button multiple times; each time it adds 10 minutes up from 10 to 60 minutes.

- The junction of left&right point lamp becomes the healing point.
An appropriate distance from the safety net is about 25cm.
- All SETS can be moved using the sliding bar to a proper healing point (except for the HEAD SET.)
- The cure hole cover in the mattress should be fit into the cure hole usually and pulled out during use.

- ④ When the time is over, the TR-7000 system stops and sounds the alarm automatically.

ATTENTION

When using the HEAD SET, please be careful so as to ensure that your hair does not get caught inside.

(3) Precautions

- ① Once it is turned on, it takes about 3 mins to preheat.
- ② Before use, read carefully and understand the 'How to cure' (P7-8) and 'Attention/Warning' (P9~10) sections.
- ③ The DISK USED HOUR indicates the total usage time, contact us to have the disk replaced before it reaches 3,000 hrs.

(4) Specification

| | |
|--|-------------------------------------|
| Electric Power Source | AC220Volt,110Volt / 50Hz, 60Hz |
| Electric Power Consumption | 230W \pm 10% \times 2 |
| Disk Temperature | Avg. 320°C(MAX 380°C) |
| Wavelength Range | 2~25 micron |
| Disk Durability | 3,000 Hrs. |
| Types of Disk | GOLD · BLACK |
| External Size(W \times L \times H) | 2,000 \times 700 \times 1,450mm |

Things to check prior to requesting A/S

If any of the following have occurred, the device is still in proper working condition.

| Error | Check point | Course of action |
|--|---|--|
| The device does not operate. | <ul style="list-style-type: none"> ⦿ Did you check that the power cord is plugged in properly to the electric outlet? ⦿ Is electrical outlet working properly? | <ul style="list-style-type: none"> ⦿ Plug the power cord into the electric outlet properly. ⦿ Plug in another product to check whether electricity is coming through your wall outlet. |
| The product emits an odor. | <ul style="list-style-type: none"> ⦿ There may be a smell when you first buy it. The smell naturally disappears after you have used it for about 3 days. (It's like the smell of furniture when first purchased.) ⦿ If foreign particles fall inside the product due to careless use, an odor may result so please take precaution. | |
| It continues to run even after the STOP button has been pressed. | <p>Pressing the STOP button during operation will automatically start up the fan and lamp for 90 seconds in order to protect the radiation plate and the case. This is not a malfunction so feel free to use without worrying.</p> | |
| It makes a strange noise. | <p>The device may make a sound if the product is at an incline or if it is directly up against the wall or other surface.</p> | |

Warranty

We warrant our goods as follows:

Please have the date of purchase written on the warranty and keep it in a secure place as you have to present the warranty to get good service from us when asking for repair.

1. This product is manufactured under strict quality control and inspection processes.
2. When this product goes out of order under the normal use of customer, we will repair it free of charge during the warranty period.
3. The damages(defaults) prescribed below are NOT to be covered by warranty.
 - Users faults by lack of care.
 - Unauthorised electric currency connection defaults.
 - Faults by the users own intention of dismantle or repairs.
 - Damages caused by natural disaster. (Fire, Flood, Earthquake, Lighting, etc.)
 - Replacement to new parts.
4. Please show us the warranty when you ask for repair service.
5. Please keep the warranty securely as it is not reissued.

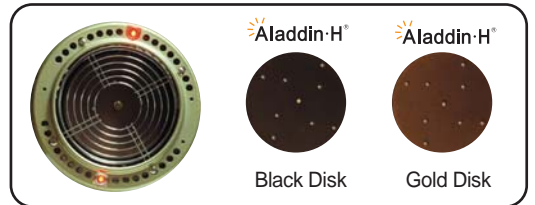
Shape or circuit of the product are subject to change without any notice to improve the quality of product.

| | | | |
|-------------------|------------------------|-----------|--|
| Customer Name: | | Tel. No.: | |
| Product Model: | Serial No.: | | |
| Date Of Purchase: | Invoice / Receipt No.: | | |
| Dealer's Name: | | Tel. No.: | |

TAERIM MEDICAL 

Aladdin·H[®]

Product information (For Home)



We invented our products which have different models to be used by various.



TR-300



DS-777



DS-777/TR-1000



DS-777Combi
(DS-777+TR-300)